

## Instructions for applying 'little' ear tags to piglets

Our advice: within 1 to 10 days after birth

### Placing the ear tag in the applicator:

Position the female part of the ear tag underneath the holder and push it all the way back

**Use the MS Little Ear Tag Applicator with spring mechanism**



**Insert the female part of the ear tag**



**The hole of the ear tag should be in the centre**




Place the male part upon the perforation pin and push it till the end

**Male part**



**Press firmly to prevent detachment**


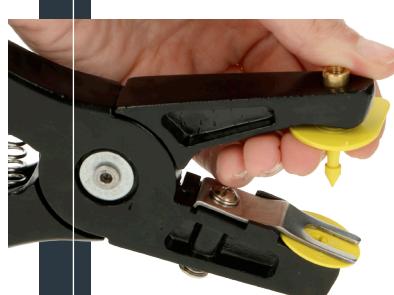


**Test the operation of the applicator**




\* If you have exchanged the piercing pin make sure to check the operation of the applicator before you start applying ear tags


# Placing the ear tag at the right spot:

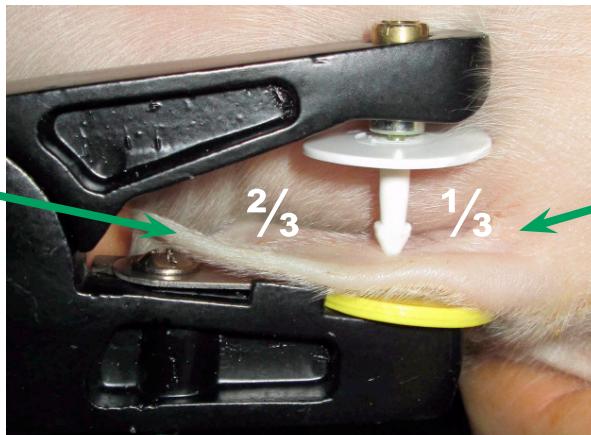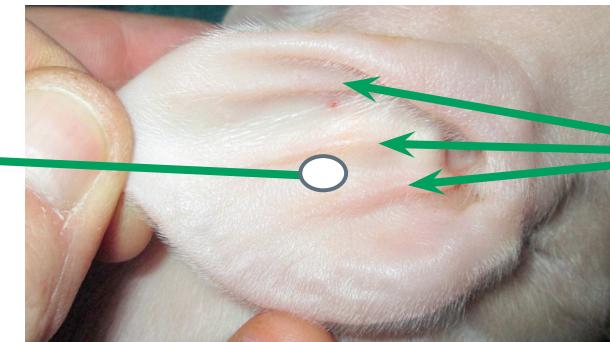


## Apply the male part from the backside of the ear

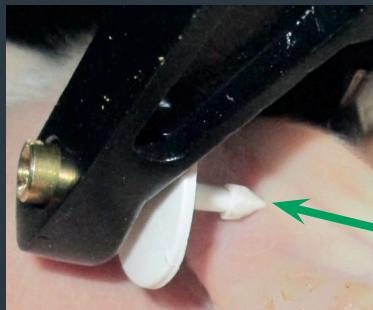

By pulling the tip of the ear the cartilage will become visible

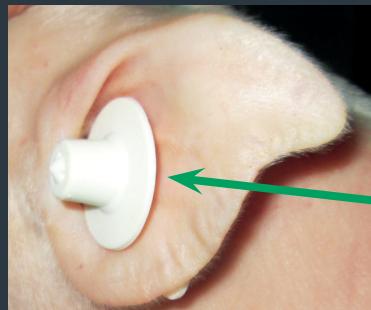

Placement between the cartilage

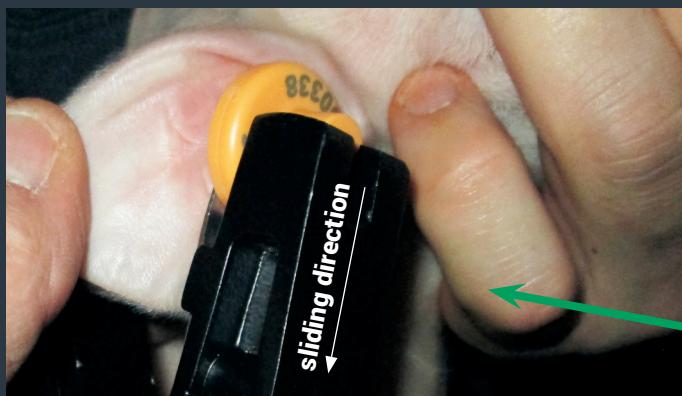

Allow the applicator to fully slide back into its free position.

Hold the ear and gently release the applicator from the female part.