

Technovit-2-Bond



The fast, 2-part PU adhesive for hoof treatment!

KERBL

Technovit-2-Bond is a quick-drying, two-part adhesive for treating hooves. Just a small amount of adhesive is all that's needed to glue wood or rubber blocks to healthy hooves in seconds in order to take the strain off diseased or damaged ones and therefore accelerate healing.



Here's how simple it is to use:

- Remove the seal on the cartridge and connect up the mixing tip.
- Eject a small amount of adhesive. This must be done so that the two components mix as effectively as possible.
- Now apply a bead of adhesive onto the block.
- Place the block quickly onto the pre-prepared, clean and dry, healthy hoof.
- After around 30 seconds, the adhesive will have set, and after around 2-3 minutes, the hoof can weight-bear again.

The benefits at a glance:

- **dries within 30 seconds**
- **bonds powerfully and permanently to wood and rubber blocks**
- **easy to use - no mixing, no waste**
- **odourless and safe to use, even in confined spaces**

ref. no.	description			€
16241	Technovit-2-Bond cartridge, 160 ml	1/10	1,200	
16242	Dispensing gun	1/10	250	
16243	Mixing tips, 10 x	1/10/240	1,920	