0609232

Cowslips (NL, DU, GB) Sabottines (F)

Schippers Europe BV
Rond deel 12
5531 AH BLADEL
Tel: +31(0)497-339771
Fax: +31(0)497-382096
sales@schippers.nl
www.schippers.nl

MS SCHIPPERS
Passion for farming

|

|

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Cowslips

Directions for use

Apply cowslips in open and well ventilated spaces only. Use chemical resistant gloves. In the event of spillage rinse with plenty of cold water.

Cowslips are orthopaedic shoes which have been specially developed to treat lameness in cattle. By applying a cowslip to the healthy claw, the cow's weight is taken away from the damaged claw. This not only takes away the pain but it also stimulates the healing process. The cowslips are kept in place by a specially developed methylmethacrylate resin.

Pre-Treatment of the claw

To ensure good attachment of the cowslip it is essential to clean the healthy claw. It may be necessary to cut the claw to achieve a good fit of the cowslip. Dead horn tissue should be removed and the horn of the sole should be smoothed with a hoof knife, rasp or a steel brush and dried with acetone or methylated spirit.

Mixing the resin

The resin can be mixed in the shoe of the cowslip. Just enough liquid should be mixed with a bag of powder to dissolve the powder (approximately 17 ml of methylmethacrylate liquid).

The mixture will become sticky very quickly and can then be applied to the claw. Tap the shoe gently in to place with a hammer or by hand and make sure that the shoe is placed horizontally and firmly to the sole of the claw. The resin should only make contact with the horn tissue, not with the skin. The drying process can be accelerated by decreasing the amount of liquid, or by warming the bottle of liquid in hot water. The shape of the shoe can be altered if necessary by heating it in boiling water but the shoe must be completely dry before it is applied to the claw.

Cowslips normally stay in place for 3-4 weeks, after which they will release themselves.

In some cases the cowslip may stay in place for 4-5 months without adverse consequences for the cow but if this occurs the claw must be checked regularly. The cowslip can be removed by splitting the shoe by means of a hoof knife and then peeling the two halves away from the claw.

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Schippers GmbH