



Functional hoof care includes appropriate treatment in the case of bovine hooves deteriorating. One of the most popular methods is immobilising the injured hoof by raising the healthy ones.

With Walkease – a new and patent-pending product – hoof care can be done even quicker and more simply than with previous systems!

Walkease blocks are made from a specially-formulated ethylene-vinyl acetate (EVA) material. They are attached to the hoof by use of a cyanoacrylate adhesive. Walkease blocks are low-cost and easy to apply. Special skill and experience are not required.

Walkease blocks are intended for use to repair minor hoof injuries. They are ideal for the recovery of such ailments as thin soles, and other conditions needing about 2-3 weeks of elevation. For serious injuries needing long-term elevation, other systems should be used.

Walkease blocks do not generally need to be removed. They ablate naturally after 2-3 weeks. It is always advisable to regularly check on the healing process and the state of the block at the same time.

Walkease blocks are available in four sizes:


small = red

medium = yellow

large = blue

large chamfered (wedgie) = white

The white 'wedgie' blocks are especially suitable for surfaces with a high abrasion factor, or for cows that would quickly 'wear down' the back of a flat Walkease block when moving. Incorrect positioning due to worn blocks can lead to overloading, which can cause inflammation.

ref. no.	description		€
16481	Walkease hoof treatment, medium, pack of 10	1/9	69,95
16482	Walkease hoof treatment, large, pack of 10	1/9	69,95
16489	Walkease hoof treatment, mixed (3xS/3xM/4xL)	1/9	69,95
16483	Walkease hoof block replacement, small, pack of 10	1/9	45,95
16484	Walkease hoof block replacement, medium, pack of 10	1/9	45,95
16485	Walkease hoof block replacement, large, pack of 10	1/9	45,95
16488	Replacement Hoof Block Walkease „Wedgy“, angled, large, pack of 10	1/9	45,95
16486	Replacement adhesive for Walkease 20 ml	1/9	21,95
16487	Hoof Rasp Shoof	1/9	9,95

Your specialized dealer:

Use: Wear gloves during this procedure to protect your hands. Avoid contact to eyes or hair with the glue. Important: the surface of the healthy claw must be treated and must have a clean, dry condition.

Note: Walkease glue is different to most other hoof-block adhesives. The natural moisture in the hoof and in the atmosphere is required to activate the glue. The hoof must not be artificially dried (hot air or alcohol).

In very cold weather, when the air is very dry, more time might be required to activate the glue. Tipp: Breathing closely on the glued surface a few times before application to the hoof will help.

For best life the glue should be stored in a refrigerator, before and after opening. The glue is best used at about room or body temperature. Taking from the fridge and carrying in your pocket before use, is ideal. Put back in the fridge after use (for longest life). After the block is glued at the claw, we recommend to cut the exceeding parts of the block!

1. Treat the injured claw as required.



2. Use the Walkease hoof rasp to prepare the healthy claw for gluing.



3. Do not use excessive glue (2 ml only) and press the block firmly onto the healthy claw. The block should overlap the hoof wall by 2 or 3 mm.



4. After about 10-15 seconds of steady pressure you will feel the block 'grab'.



5. Drop the leg for the cow to put weight on the block quickly. This is most important for attach the block.

