

0609966

250 ml

MS
GLUEPRO

MS Schippers



**Application
time:**
30 sec.



**Application
temperature:**
30°C max.
21°C min.

How to use MS GluePro:

1. Treat the injured claw
2. Trim the sound claw as flat, clean and dry as possible
3. Place the MS GluePro cartridge into the applicator gun
4. Remove cap from cartridge and apply appropriate size mixing tip
5. Hold the cartridge vertically and press a small amount of glue out, so air bubbles are removed
6. Apply MS GluePro adhesive to wooden block
7. Apply block immediately to prepared sound claw
8. Hold firmly for 30 seconds so a layer of app 3 mm of glue remains between claw and block
9. Let cow stand on block 2 to 3 minutes before releasing it.

Helpful MS GluePro Tips:

- Make sure cartridge openings are clear (free of any cross contaminated or cured product prior to screwing on a new mix tip.
- Mixing tip may be left on cartridge until next use or mixing tip may be replaced with cap

Net weight 270 g

Schippers Europe BV
Smaragdweg 60
5527 LB Hapert
Tel: +31(0)497-339 779

Schippers France Sarl
ZA de la Retaudais - CS 10009
35137 BEDEE (FR)
Tel: 02 99 61 40 40

Schippers GmbH
Kölner Straße 62
47647 Kerken (D)
Tel: 02833-92360

Schippers BV
Grens 114
2370 Arendonk (BE)
Tel: 014-672356


Schippers Agricola SL
Ctra. Montmeló 76
08403 Granollers
Tel. 935 689 128

Schippers Export BV
Smaragdweg 60
5527 LB Hapert
Tel: +31(0)497-339774

Component A

Hazard statements (CLP)

: Suspected of causing cancer. Causes serious eye irritation. May cause allergy or asthma symptoms or breathing difficulties if inhaled. Causes skin irritation. May cause an allergic skin reaction. May cause damage to organs through prolonged or repeated exposure. May cause respiratory irritation.

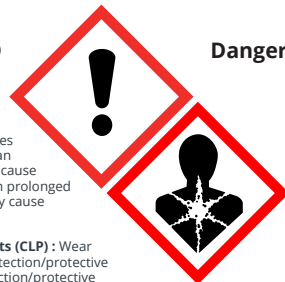

Danger

Precautionary statements (CLP)

: Wear protective gloves/face protection/protective clothing/respiratory protection/protective footwear. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Get medical advice/attention. IF experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician. Dispose of contents/container in accordance with regulations on hazardous waste or packaging and packaging waste respectively. Contains isocyanates. May produce an allergic reaction. As from 24 August 2023 adequate training is required before industrial or professional use

Contains: 4,4'-methylenediphenyl diisocyanate

Storage: Do not allow to freeze. Store and use at room temperature (above 21°C). Cold product needs adequate time to warm core temperature.

UFI: 1H30-A0QE-Q00C-TVC4

Component B

Hazard statements (CLP)

: Causes serious eye irritation. Causes skin irritation. May cause respiratory irritation.



Warning

Precautionary statements

(CLP) : Wear protective gloves/protective clothing/respiratory protection/eye protection/protective footwear. IF ON SKIN: Wash with plenty of water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container in accordance with regulations on hazardous waste or packaging and packaging waste respectively.

Contains: Polyether Polyol

Batch N°:



UFI: QM30-U0DU-000V-F6X6

Made in the Netherlands

25 LV 02

www.schippers.eu

TheSchippersGroup
Let's beat AMR with HyCare

Download app!
Scan this product



0609966