

How to use GLOOF:

1. Treat the injured claw
2. Trim the sound claw as flat, clean and dry as possible
3. Place the GLOOF cartridge into the applicator gun
4. Remove cap from cartridge and apply appropriate size mixing tip
5. Hold the cartridge vertically and press a small amount of glue out, so air bubbles are removed
6. Apply GLOOF adhesive to wooden block
7. Apply block immediately to prepared sound claw
8. Hold firmly for 30 seconds so a layer of app 3 mm of glue remains between claw and block
9. Let cow stand on block 2 to 3 minutes before releasing it.

Helpful GLOOF:

- Make sure cartridge openings are clear (free of any cross contaminated or cured product) prior to screwing on a new mix tip.
- Mixing tip may be left on cartridge until next use or mixing tip may be replaced with cap

Storage

Do not allow to freeze. Store and use at room temperature (above 21°C). Cold product needs adequate time to warm core temperature.

Net weight
270 g

Europe:

Schippers Europe BV
Smaragdweg 60
5527 LB Hapert
Tel: +31(0)497-339 779

France:

Schippers France Sarl
ZA de la Retaudais -
CS 10009
35137 BEDEE (FR)
Tél: 02 99 61 40 40

España:

Schippers Agrícola SL
Ctra. Montmeló 76
08403 Granollers
Barcelona - España
Tel. 935 689 128

Deutschland:

Schippers GmbH
Kölner Straße 62
47647 Kerken (D)
Tel: 02833-92360

België:

Schippers BVBA
Grens 114
2370 Arendonk (BE)
Tel: 014-672356

Export:

Schippers Export BV
Smaragdweg 60
5527 LB Hapert
Tel: +31(0)497-339774

250 ml

MS GLOOF

0609967

Component A

Hazard statements (CLP)

: H315 - Causes skin irritation. H317 - May cause an allergic skin reaction. H319 - Causes serious eye irritation. H332 - Harmful if inhaled. H334 - May cause allergy or asthma symptoms or breathing difficulties if inhaled. H335 - May cause respiratory irritation. H351 - Suspected of causing cancer. H373 - May cause damage to organs through prolonged or repeated exposure.

Precautionary statements (CLP) : P260 - Do not breathe dust, fume, gas, mist, vapours, spray. P280 - Wear protective clothing, eye protection, face protection, protective gloves, hearing protection. P284 - Wear respiratory protection. P305+P351+P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P405 - Store locked up. P501 - Dispose of Contents and container, to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

Contains: 4,4'-METHYLENEDI(Phenyl Diisocyanate), OLIGOMERS; Polyisocyanate prepolymer based on mdi

UFI : 3SWE-259U-N00U-UY9H

Component B

Hazard statements (CLP)

: H315 - Causes skin irritation. H317 - May cause an allergic skin reaction. H319 - Causes serious eye irritation. H411 - Toxic to aquatic life with long lasting effects.

Precautionary statements


(CLP) : P261 - Avoid breathing dust, fume, gas, mist, spray, vapours. P273 - Avoid release to the environment. P280 - Wear protective gloves, protective clothing, eye protection, face protection, hearing protection. P305+P351+P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P333+P313 - If skin irritation or rash occurs: Get medical advice/attention. P501 - Dispose of Contents and container, to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

Contains: 6-METHYL-2,4-BIS(METHYLTHIO)PHENYLENE-1,3-DIAMINE; M-PHENYLENEBIS (METHYLAMINE)


UFI : GT8G-PR0W-X10W-2C2J

Made in the Netherlands

Danger

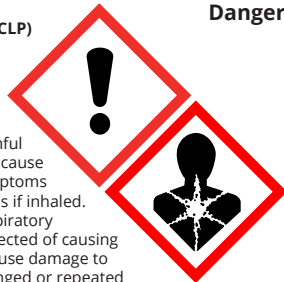

Warning

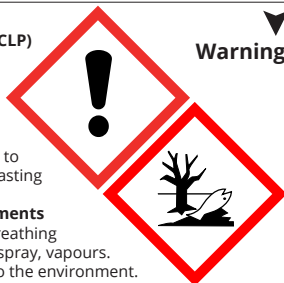

TheSchippersGroup

Let's beat AMR with HyCare

Download app!
Scan this product



0609967

21 LV 06