BANDE VAGINALE DE BÜHNER

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
Veterinary use only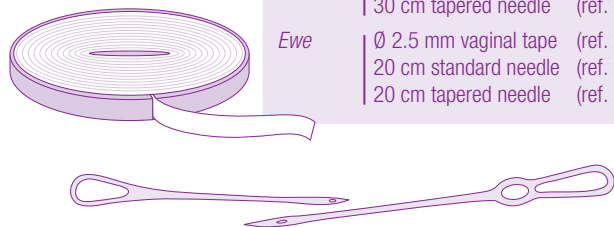

Which product to use ?

<i>Cow</i>	Ø 6.5 mm vaginal tape	(ref. 220850)
	30 cm standard needle	(ref. 227900)
	30 cm tapered needle	(ref. 227901)
<i>Ewe</i>	Ø 2.5 mm vaginal tape	(ref. 220852)
	20 cm standard needle	(ref. 227902)
	20 cm tapered needle	(ref. 227903)

The Bühner vaginal tape is used to suture the vulva when treating a prolapsus, whether related or not to a condition of gestation of the animal.

It permits a surgical treatment requiring simple, inexpensive and reliable equipment. It eliminates the disadvantages of pins which sometimes get loose, causing vulvar wounds and resulting in an immediate recurrence of the problem.





The Bühner method causes little trauma and permits a solid as well as esthetic suture since it is virtually invisible. Furthermore, it provides a high percentage of success as it involves no post-surgical risks.

The Bühner method should be used in connection with a medical and hygiene treatment.

The Bühner vaginal tape is inserted subcutaneously around the vulva with an adapted needle, 30 cm long for a cow and 20 cm long for a ewe.

Instructions for use

After reducing the vaginal prolapsus, retention of the vagina is required during 2 to 4 weeks in order to hold it in place and to prevent any recurrence.

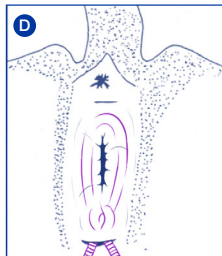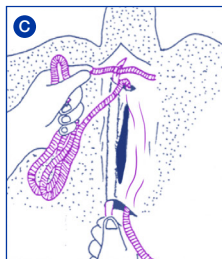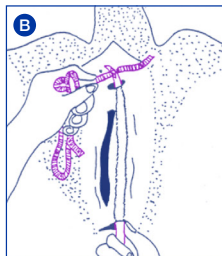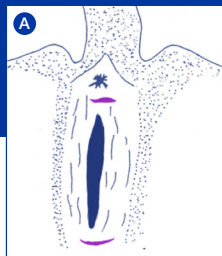génia

The operation is always performed with the animal standing up.

Following a low epidural anesthesia and reduction of the prolapsed mass, two horizontal incisions are cut in the skin of the vulva labia. A first incision, approximately 1 cm long, is done midway between the upper commissure of the vulva and the anus. The second one, approximately 2 cm long, is cut just below the ventral commissure **A**.

The needle is inserted through the incision at the ventral commissure and is run parallel along one side of the vulva through the perivaginal connective tissue. It is pushed backwards and fairly deep, until it reappears out of the incision cut at the upper commissure of the vulva. The cord is then threaded through the eye of the needle which will be pulled back and out of the lower incision **B**.

The same operation is repeated in the same manner on the other side of the vulva, in such a way that the cord will entirely surround the vulva **C**.

Both loose ends of the Bühner cord at the ventral commissure are tied and tightened moderately, just enough to let 3 or 4 fingers pass through. Once the suture is finished, the knot is placed underneath the skin to obtain a purse string suture **D**.

The upper incision may be stitched up and the suture can remain in place for several months without any risk whatsoever.

This method can be used without any restriction with gestating females prone to vaginal inversions. In such cases, the Bühner suture should be removed when the second phase of the calving or lambing begins.

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