MS Gasstaartcoupeerder Super
MS GasSchwanzkupierer Super
MS Coupe queue Super
MS Gas operated tail docker Super
MS Cortador de rabos a gas súper

0905208

NL	<i>MS Gasstaartcoupeerder Super</i>	
	<i>Gebruiksaanwijzing.....</i>	<i>3</i>
DE	<i>MS GasSchwanzkupierer Super</i>	
	<i>Gebrauchsanweisung.....</i>	<i>5</i>
FR	<i>MS Coupe queue Super</i>	
	<i>Mode d'emploi.....</i>	<i>6</i>
EN	<i>MS gas operated tail docker Super</i>	
	<i>Manual.....</i>	<i>7</i>
ES	<i>MS Cortador de rabos a gas súper</i>	
	<i>Instrucciones para el uso.....</i>	<i>9</i>

MS Gasstaartcoupeerder Super

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MS Gas operated tail docker Super

A device for painless and effective docking the tails of new-born piglets.

Cuts and cauterises at the same time, to reduce the risk of infection.

Economical: a cylinder full of gas will be sufficient for approximately 120 piglets.

Built-in electronic ignition.

Operating instructions

Temperature: Approx 400°C

Burning time: Approx 1 - 1½ hours.

It is recommended to refill the gas cylinder before each use in order to avoid running out unexpectedly. In normal use, you should expect to be able to treat approximately 10 litters with one full cylinder. A clear window at the back of the cylinder enables a visual check that it contains gas.

Refilling

Refilling with a gas cigarette lighter refill cartridge is easy.

Do not smoke while carrying out this operation.

Select an adaptor to suit if necessary. Hold the Tail Docker with the filling end upwards, and invert the refill cartridge, fitting it to the filling point in the base. To purge air from the cylinder, open the gas valve by pushing it towards the blade (Position "I") and press the two cylinders together. Once gas has come through the burner end (you will notice a slight reddening of the blade), close the gas valve (Position "O"), and continue to apply pressure until gas escapes from the filling nozzle. We recommend that after filling you stick a small piece of adhesive tape over the end, as it is possible for any dirt which gets in there to be blown inside the gas cylinder when refilling, and this can eventually block the valve at the burner end and stop the burner from working.

The temperature can be adjusted with the rotary knob at the back of the cylinder.

In a couple of minutes, the blade will have reached operating temperature (Approx 400°C), and the tail docker is ready for use. To save fuel, you can turn down the burner between litters.

It is the heat of the blade which does the work, not the sharpness. Cut gently until the tail falls off. Don't apply excessive pressure, as this can cause bleeding, which may result in infection. It is important not to cut the tail too short, as this can damage the nerves of the tail. If there is no sensation in the tail, the animal is more likely to be the victim of any future tail-biting. When you have finished using the device turn it off by returning the slider to the "O" position, taking care that the blade will remain hot for a while.

Changing the blade

Undo the screw from the metal ring to enable the blade to be removed and replaced.

Caution!

To clean the blade use a dry cloth, or possible some fine sand-paper. The gas cylinder contains gas under pressure. Do not expose it to temperatures over 50°C or to direct sunlight.

Trouble Shooter

If the MS Tail Docker does not work well, it could be that there is air in the gas cylinder. This can be caused by repeated re-filling, when a little air gets in each time.

The solution is simple:

- 1. Hold the Tail Docker with the filling point upwards, and press on the refill nozzle with a small object.*
- 2. Continue to hold it in until liquid gas escapes.*
- 3. Allow it to return to ambient temperature (as gas escapes, it will become cold)*
- 4. Once it has regained ambient temperature, refill with gas again.*

NB. Always hold the Tail Docker blade downwards when refilling.

If the blade becomes loose, remove the burner and cylinder from the chassis, light it, and let it warm up for several minutes until the metal parts are hot. Turn off the burner, and re-tighten the screw in the fixing ring. When it cools down, it will no longer move.

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Schippers Italia Srl
Schippers UK Ltd.
Schippers Schweiz GmbH