MS Smart taildoker incl. ventilator

0909994

NL	MS Smartcoupeerder incl. ventilator	
	Gebruiksaanwijzing	3
EN	MS Smart taildoker incl. ventilator	
	Manual.....	8
DE	MS Wärmematte/Regler	
	Gebrauchsanweisung.....	13
FR	MS Plaque chauffante/ Boîtier réglage	
	Mode d'emploi.....	18
ES	MS Placas calefactoras/ controlador	
	Instrucciones para el uso	23
IT	MS Tappeto riscaldante/ Combinatore	
	Manuale di istruzioni.....	28
DK	MS Tappeto riscaldante/ Combinatore	
	Manuale di istruzioni.....	33

MS Smartcoupeerder incl. ventilator

Introductie

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MS Smart taildoker incl. ventilator

Introduction

This manual provides instructions that will guarantee proper installation, operation and maintenance of the Smart Tail Docker with Fan (Forthwith known as the Smart Tail Docker). You should only perform installation after carefully reading and understanding this manual. Improper use of the installation could cause damage to the installation, yourself, or goods and materials. Therefore, only use the installation in technically perfect condition, and only for the purpose for which it was designed.

No technical changes to the installation are permitted, unless the manufacturer has given express written permission. Without the abovementioned permission the manufacturer's guarantee becomes void. This guide contains information regarding operation, maintenance and tracing and troubleshooting faults. You should take note of all relevant information in this manual before installing or operating the system. This is important for averting material damage and personal injury. Careful and precise assembly results in good operation.

In the design of this installation, MS Schippers has given special attention to safety during installation, operation and maintenance. However, it remains the user's responsibility to observe the safety regulations at work.

Application

The Smart Tail Docker module with fan is used on the MS Multi Treatment Unit, to dock new born piglets' tails. Due to the heat of the knife, the wound is immediately cauterized, reducing the risk of infection. The use of the MS Smart Tail Docker has a positive effect on animal health (wound cauterized more quickly and effectively).

In addition, the Smart Tail Docker module is equipped with a spacer, which can be adjusted to dock all tails at the same length.

Provisions

MS Schippers assumes no responsibility for errors contained herein or for any direct or indirect damages or consequential damages of any nature whatsoever arising from or in connection with provision, function or use of this document.

MS Schippers reserves the right at any time, without prior notice, to make changes to the structure of the Smart Tail Docker or the information in the manual, without any resulting obligation to apply these changes to products already delivered.

In case of misuse, the manufacturer's guarantee and responsibility becomes void.

Contents

Introduction	8
Contents	9
1. Technical specifications	9
2. Precautions and safety instructions	9
3. Smart Tail Docker Module with Fan Parts	10
4. Mounting on the MS Multi Treatment Unit	10
5. Operation and use	11
6. Maintenance	11
7. Faults and repairs	12

1. Technical specifications

Manufactured and designed in compliance with: EN 61558-2-6

Type: TD01 MS MK-3	Ta: 40
VA: 100	IP: 65
Primary: 230 V	Hz: 50/60
Secondary: 1 V – 100 A	Fuse/breaker: 1.6 A

Dimensions: (outside dimensions)
190 x 420 x 370 mm (W x L x H)

2. Precautions and safety instructions

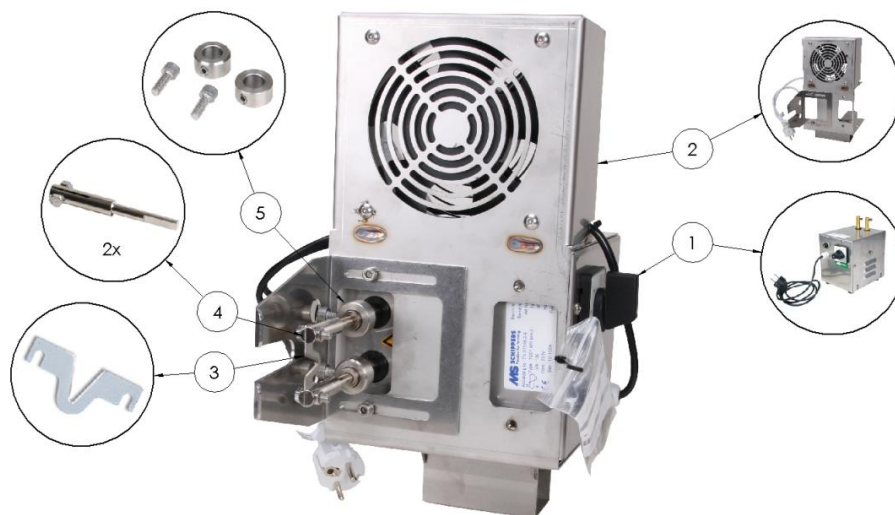
The Smart Tail Docker complies with safety regulations applicable within the European Union.

The following symbols are used in this manual, to ensure proper and safe use of the Smart Tail Docker.

	This symbol indicates a tip.
	This symbol indicates a general point for attention.
	This symbol indicates a situation that requires extra attention to prevent direct contact with parts that may carry a live electrical current.
	This symbol indicates a situation that requires extra attention to avoid direct contact with hot surfaces.

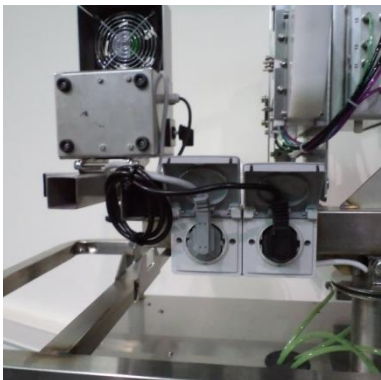
3. Smart Tail Docker Module with Fan Parts

No.	Article number:	Description: (EN)
1.	0909999	MS Smart Tail Docker excl. Blade
2.	1406544	Smart Tail Docker Fan Angle Support
3.	0909982-7	Set 7 pcs. inlay V-knife for MS Smart Tail Docker
4.	0909981	M5 Adaptor for inlay knives
5.	0909983	Smart Tail Docker chuck recon set, S.S.

4. Mounting on the MS Multi Treatment Unit

The MS Smart Tail Docker module is intended for use on the MS Multi Treatment Unit. The following describes how to install it.

1.	2.	3.
		
Mount the Smart Tail Docker, as shown, onto the combi-support on the MS Multi Treatment Unit.	Insert the electrical plugs into the appropriate supply outlets. If the sockets are live, the fan will run immediately.	When moving the Multi Treatment Unit, you can tilt the MS Smart Tail Docker to avert the risk of damage with it protruding from the unit.

Electrical safety of the equipment is only guaranteed when it is connected to a grounded supply outlet.

5. Operation and use

Do not leave the Smart Tail Docker in animal housing longer than necessary. Ensure clean and dry storage.

NOTE! The cutting knife and extension tube are hot during operation. Always take precautions to avoid contact with these surfaces.

To ensure cauterising the wound as well as possible, it is best to dock the tail in a calm manner, starting from underneath.

The MS Smart Tail Docker has a heat control switch at the front. This control switch allows the user to adjust the power to 3 different heat settings. This is due to the different resistance levels of the docking knives. The user should adjust the setting of this switch until the knives give an orange glow. If the knives give a white glow, the setting is too high. This will shorten the service life of the knives.

When fitting the knives, the user should take care to ensure that the fixing screws are tightened properly, as this improves the contact resistance.

The service life of the knives depends largely on the operating conditions, e.g. working speed, pressure applied, temperature, age of piglets. Testing has yielded results varying from treatment of 40 to 400 litters.

6. Maintenance

The Smart Tail Docker can be dismantled from the MS Multi Treatment Unit and put onto a workbench for cleaning and maintenance. When doing this, ensure that the MS Smart Tail Docker is not exposed to water or dampness. You can clean the equipment with compressed air or a dry cloth.

Always ensure that the system has been disconnected before replacing the knife.

If the knife fails to reach an adequate heat, this may indicate that the connection between the knife insert and the adapter bushing needs cleaning (wire brush/sandpaper).

If the screws are not tightened properly, this can also result in the knife producing less heat.

The screws will need to be tightened occasionally due to materials expanding and contracting. Note: tight means tight!

Make sure to use tools when the knife insert needs changing, as the parts will all be hot!

Make sure you have two sets of adapter bushes. This will allow you to fit one set while allowing the other to cool down.

7. Faults and repairs

Contact MS Schippers customer service.
Customer service department
Tel: +31(0)497-339787

klantenservice.nl@schippers.eu

Contact address

P.O. Box 122
5530 AC Bladel
The Netherlands

- Status of your delivery
- Request for Guarantee
- Repairs
- Returns
- Complaints

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**Schippers Italia Srl**
**Schippers Schweiz GmbH**
**Schippers UK Ltd.**
**LLC Schippers Russia**
**Schippers Canada Ltd**
7102 52nd Street Bay # 18 • Lacombe, AB • T4L 1Y9