MS Schippers

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EN MS Smart Taildoker

The MS Smart Tail Docker is a stationary docker, used for docking the tails of newborn piglets. Due to the heat of the knife, the wound is immediately cauterized, reducing the risk of infection.

The use of the MS Smart Tail Docker has a positive effect on animal health (wound cauterized more quickly and effectively).

The standard 220Volt appliance is supplied without the V-knife.

The MS Smart Tail Docker has a heat control switch at the front. This control switch allows the user to adjust the power to 3 different heat settings. This is due to the different resistance levels of the docking knives. The user should adjust the setting of this switch until the knives give an orange glow. If the knives give a white glow, then the setting is too high which will shorten the service life of the knives.

When fitting the knives, the user should take care to ensure that the fixing screws are tightened properly, as this improves the contact resistance.

Maintenance

If the knife fails to reach an adequate heat, this may indicate that the connection between the knife insert and the adapter bushing needs cleaning (wire brush/sandpaper).

The screws will need to be tightened occasionally due to the materials expanding and contracting. Note: tight means tight!

Use the stainless steel service kit (0909983, see parts list) if the threads on the pins in the knife are damaged.

If the screws are not tightened properly, this can also result in the knife producing less heat.

Make sure to use tools when the knife insert needs changing, as the parts will all be hot!

Tip: Make sure you have two sets of adapter bushings. This will allow you to fit one set while allowing the other to cool down.

The service life of the knives depends largely on the operating conditions, e.g. working speed, pressure applied, temperature, age of piglets. Testing has yielded results varying from treatment of 40 to 400 litters.

It is recommended that you use the spacer 3408920 in order to guarantee a uniform tail length.

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© Copyright 2014