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Caution: syringes are suitable only for use with animals

How to avoid the need for repairs!

The syringe should be cleaned after each use. Before cleaning the syringe, all seals and rubber plungers should be removed in order to ensure a long service life. Cleaning agents which contain vinegar or citric acid should not be used. For easier running of the rubber plunger, it should be lubricated with silicone after cleaning.

Tensioning the rubber plunger

1. Push plunger rod forward as far it will go.
2. Push handle parts together so that the rubber plunger latches in the glass cylinder.
3. Turn tensioning knob clockwise until slight resistance is noticed.

Setting the dose

With HAUPTNER-MUTO syringes it is possible to inject different-sized doses. The desired dose is selected by moving the adjustment sleeve. It must be ensured that the pin latches in the groove corresponding to the desired dose.

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Injection stick, made of flexible plastic, total length 70 cm. For use in connection with automatic injection syringes.

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Injection stick, made of flexible plastic, total length 90 cm.

For use in connection with automatic injection syringes.

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Injection extension for Muto-Express II automatic syringes and Muto®-syringes (after removing the adapter ring), length 35 cm.

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Injection extension for Muto®-syringes with screw-on mount, length 25 cm, made of strong plastic material, recommended for injection of pigs, – at request also in different lengths available –.

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