1405403
Kaycee injection syringe

English
Nederlands
Deutsch

MS Schippers

Passion for Farming

🇳🇱 Klantenservice
🇩🇪 Klantenservice/Service Clientèle
🇩🇪 Kundenservice
🇪🇺 Kundeservice
🇪🇸 Servicio al cliente
🇬🇧 Customer Service
🇫🇷 Service Clientèle
🌐 Customer Service Export

+31(0) 497-339787
+32(0)14-820713
+49(0) 2833 923630
+45 89884187
+34 931816433
+44(0) 1733592049
+33(0) 299854747
+31(0) 497700278

Kaycee injection syringe

Table of contents

Design	3
Injection syringe MK 2	3
Injection syringe MK 3	3
Use of quick coupling	3
Sterilization	3
Setting of the dosage	3
After usage	3
Important parts Kaycee syringe	4
Tips for General maintenance	4
Pay attention:	4
Maintenance	5
Service	5
Repair set with spare parts for syringes	5
Injection cannula	5
Cleaning	5
Seasons	5
Plastic mounting bottle	5
Dismantle	5
Repair	5

Design

Your kaycee syringe is easy and quickly to use for precise work. The syringe is easy to maintain and has a long endurance. The entire syringe is made of high quality material and each syringe is machined by means of Digital calibration.

Injection syringe MK 2

Kaycee 2 cc syringe with a quick coupling. Variable dosage in 0.25 cc in steps, from 0.5 cc to 2 cc, luer lock needle connection

Injection syringe MK 3

Kaycee 5 cc syringe with a quick coupling. Variable dosage in 0.5 cc in steps from 1.0 cc to 5 cc, luer lock needle connection.

Use of quick coupling

Loosen the seal ring of the quick coupling. Press the seal ring downwards. Now slide a 100 cc medicine vial over the needle set. The next step is to pull the plastic ring upwards, to pin the bottleneck nice and tight. Now twist the plastic part of the quick coupling back again. The medicine bottle is now fully fixed and ready for use. To replace the medicine bottle, repeat these actions, but now in reverse order.

Sterilization

1. Remove the plastic handle by turning it counter clock wise and put it down. Don't sterilize this plastic handle.
2. Pump, pre cooked or distilled water in the spray. Wrap the syringe, hose and needles to be used in a clean white cloth and immerse it under water. Boil for 10 minutes.
3. Pump out the water and place the plastic handle back.


Setting of the dosage

Set the dosage by turning the dosage button. This dosage button clicks the exact dose with a complete turn. The dosage can be read at the end of the handle by means of a window.

After usage

Clean thoroughly. Immediately after using the syringe, Rinse it with hot water and soap, then rinse thoroughly again with water. Remove the piston after unfastening of the mounting slots on the side arms. Lubricate the o-ring with liquid medicinal paraffin or lubricant and place it back where it belongs. Never change pistons of different syringes! Each piston is specially calibrated for that one syringe.

Important parts Kaycee syringe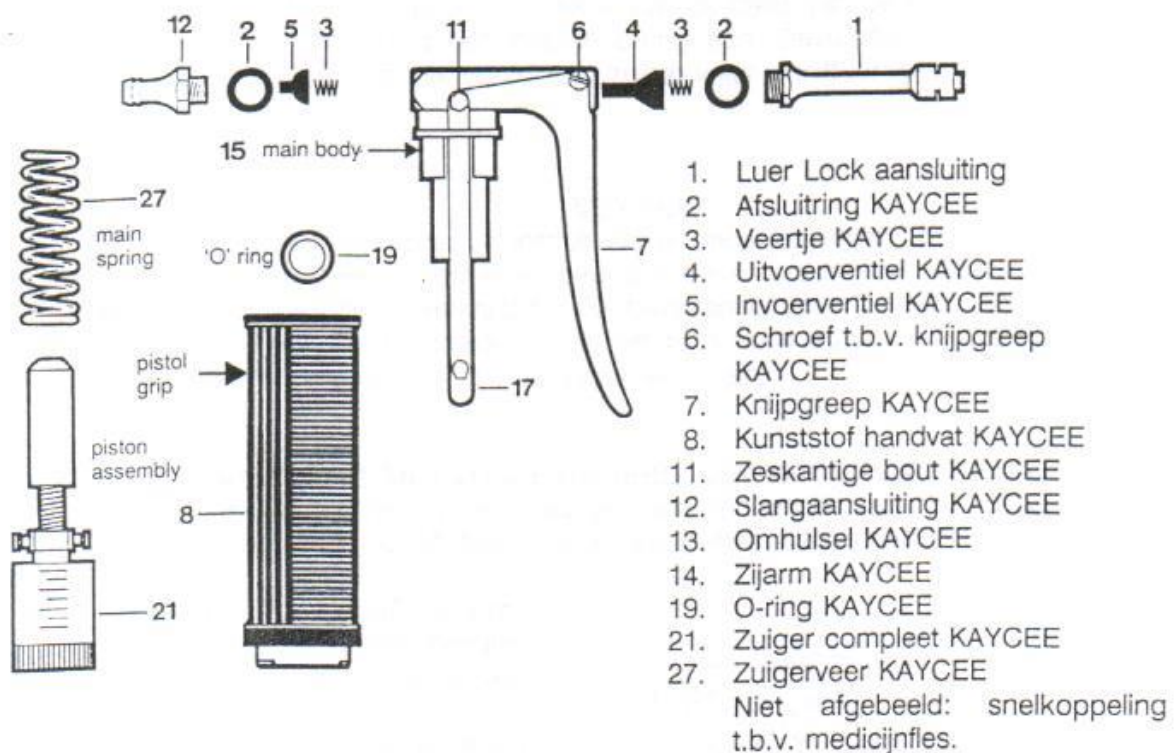

Tips for General maintenance

If the kaycee syringe doesn't dose accurately anymore, check the springs for hairs, feathers and chaff. Work over a blade to prevent loss of loose parts and remove the hose connector (12). Make sure that the valve seat in the hose connection is clean and that the semi-circular insertion valve (5) and the spring (3) are clean and not damaged. Unscrew the luer lock needle connector (1) and remove the spring with tweezers. Immediately behind the spring is the output valve (4) which can be blown out of the cover. Check for dirt and dust. Clean and reassemble again.

Pay attention:

After use and cleaning of the kaycee syringe, make sure that you keep the piston and piston spring under the handle well in order to maintain the syringe properly. You can do this by using oil or vaseline!

Make sure that the product is properly assembled. The semi-circular output valve fits into the hose connector ("tail") and the spring must be placed in the rear of the hose before the hose connection is connected. Also, the output valve fits exactly in the front of the cover ("tail" inward). Put this valve in place before attaching the spring, seal ring and a luer lid needle connector are mounted.

Do not change valves out of place! If you have more than 1 syringe, never change the pistons, make sure that each piston stays with the original syringe (pistons are individually calibrated for this particular syringe).

Maintenance

If after a long time you have a leak on the base of the handle, remove the piston and apply a new o-ring. However, it must be well lubricated with oil before the piston is replaced again.


Service

Your syringe is a precision instrument and must be checked regularly. Make sure that you keep the instructions somewhere easy to find so you can always read them when necessary. Repair sets are available. They contain springs, sealing rings and o-rings. Whilst ordering a repair set always mention the type of syringe. We can also repair and recalibrate in our workshop at Schippers Bladel B.V.

Repair set with spare parts for syringes.

Sets with input and output valves, feathers, sealing rings and o-rings are available from Schippers Bladel; For each type of syringe there is another repair kit.

- Kaycee Mk 2 Repair Kit with 1x output valve, 1x input valve, 2x seal ring, 1x o-ring and 2x spring.
- Kaycee Mk 3 Repair Kit with 1x output valve, 1x input valve, 2x seal ring and 2x spring.

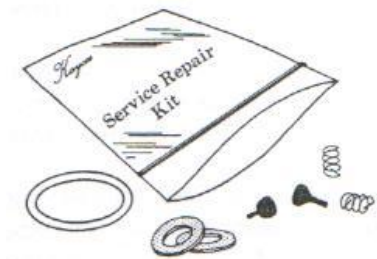Injection cannula

A disposable injection cannula with aluminum cone (luer) is great to use. However, it is very important to disassemble this properly. With a special key or key, the needle must be removed from the left. (See arrows of Vaccination stick)

Cleaning

Clean the syringe externally after use with abundant lukewarm water and soap. Rinse the plastic vial several times and then pull the syringe several times with lukewarm water from the vial.

Seasons

If the syringe is not stored freeze-free, it can freeze when it is cleaned as prescribed with plenty of water. When you handle the plastic handle of a frozen syringe with force, you will break two leverages. First defrost the spray with hot water, and then handle the plastic handle with policy.

Plastic mounting bottle

You can mount a 250ml and 500ml bottle. Do not twist the grab bar to the fullest. This causes damage to the screw of the vial, which prevents them from sealing.

Dismantle

Do not disassemble the syringe. Both the front and the back of the gun has small valves and feathers. When disassembled on the farm, these part easily fall on the ground or in the well, and the syringe won't work anymore. If the syringe has to be disassembled, be aware that during installation, the grafting stick is not tightened more firmly than "hand tight".

Repair

In case of malfunction or damage, you can put the syringe up for repair.

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