-  *Bedienungsanleitung*
-  *Instruction Manual*
-  *Manual de instrucciones*
-  *Mode d'emploi*

DRENCH – MATIC®

by **HSW**

for veterinary use only

Henke-Sass, Wolf GmbH
Keltenstrasse 1
D-78532 Tuttlingen
Germany
Tel. +49 74 62 94 66 207
Fax +49 74 62 94 66 52 08
www.henkesasswolf.de
E-mail: info@henkesasswolf.de

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DRENCH – MATIC®
by HSW

- ◆ Automatic self-filling syringe
Can be used, for example, for oral application of sheep and cattle
- ◆ Innovative and ergonomic design
- ◆ Simple handling and easy cleaning
- ◆ Plastic barrel
highly resistant adjustable dosage
- ◆ Design for use as an Injector or Drencher

The DRENCH-MATIC® is available in two different versions: 10 / 23 ml (silver) 23 / 30 ml (white)


The DRENCH – MATIC® has been designed for the oral application of sheep and cattle or for multiple dosing. The ergonomic, user – friendly design and strong materials which are used in manufacturing, guarantee a tireless operation and a high lifetime of the DRENCH – MATIC®

It is essential that the drencher is cleaned before each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.

Using a tube feeding syringe

1. Attach one end of the tube to the drencher
2. Option 1: Attach the other end of the tube to the selected Draw Off Cap
3. Option 2: Attach the other end of the tube to the bag pack
4. Loosen dose adjustment look nut (A)
5. Turn dosing adjustment screw (B) to set the required dosage. Retighten lock nut. (A)
6. Handle spring pressure can be adjusted (C) for comfort and tension

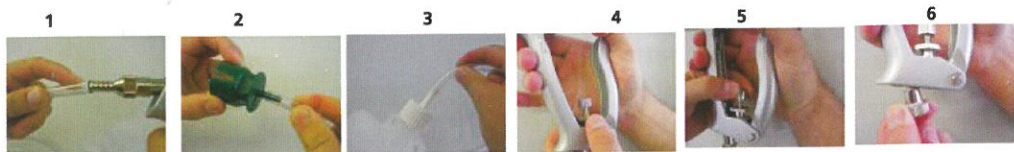
Using an Injection Syringe

For this application, additional accessories are required.

- a) Fit Luer Lock Adapter by means of the screw nut onto the tip (attached nozzle to be removed first)
- b) Insert Luer Lock needle
- c) Adjust/set dosage as described under item 4/5 by pushing the handle in forward position
- d) Squeeze adjustable handle and then release to draw fluid into the barrel.
To remove air bubbles inside the barrel, repeat as mentioned in step (c) taking care not to waste fluid
- e) In case of air bubbles repeat step (c)

Dose Setting: Align the black surface of the piston to the required dosage shown on the barrel.
The syringes are equipped with different pistons; please operate according to the following illustrations.

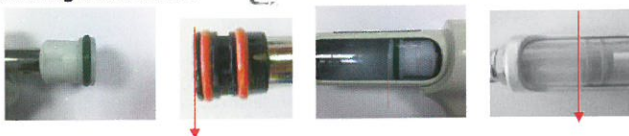Instruction / Maintenance of the HSW DRENCH – MATIC®

Remove all traces of material from drencher by flushing thoroughly with a warm water detergentmix.

Replacing piston O-ring / Cleaning of the barrel

(Varied according to each model)

Should it become necessary to replace the piston O-ring the subsequent procedure must be followed:

picture 1


picture 2

picture 3

picture 4

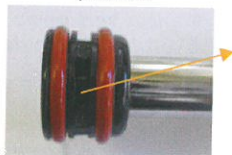- 1) Adjust the handle by means of the dosage setting to maximum as instructed on page 1 (point 4/5)
- 2) Unscrew fixing ring for the handle
- 3) Remove piston rod by sliding it backwards
- 4) Replace two O-rings and lubricate new O-rings as well as one of the notches on the piston for self-lubrication during use with 100% pure oil, cold squeezed or equivalent.
For the cleaning (boiling), remove barrel from housing by pushing it back.
Reinstall the opposite way.
- 5) Make sure that only original spare parts are used to guarantee function.

Version with pin

Version with flange

one O-Ring

23 ml/30 ml (white) with pin lock: unscrew the fixing screw on housing and turn housing by $\frac{1}{4}$ and retract the housing with barrel from the handle. You can now remove the barrel.

23 ml/30 ml (white) with flange: unscrew the fixing screw and slide the housing with barrel backwards. You can now remove the barrel.

Replacing the Valves

- unscrew fixing screw on the front part
- exchange or clean valve, spring and washer
- Version with pin or flange: Remove the valve to the front, against a slight resistance.
Unscrew tubing attachment at the rear end of the syringe or unscrew the piston from the piston rod with a screw-wrench and operate correspondingly
- exchange or clean valve, spring and washer or unscrew the piston from piston rod with a screw-wrench and operate correspondingly

With all versions, please pay attention to a correct assembly of the springs!

Desinfection

It is recommended to disinfect the DRENCH – MATIC® as well as the needles or nozzles before and after each use.

- 1) Draw "hot" water into the barrel
- 2) Suspend the complete DRENCH – MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes.
- 3) Remove the DRENCH – MATIC® from container, wrap cloth around the handle and pump it dry

TO LUBRICATE

If piston becomes sluggish: Remove piston and lubricate O-rings (according to each model) with 100% pure oil, cold squeezed or equivalent vegetable oil. Please follow the manufacturers durability and storage life instruction. Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 700037-05/12 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700037-05/12 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700037-05/12 · Reservados los derechos de modificaciones técnicas y de diseño · Impreso en Alemania
© copyright · Henke-Sass, Wolf GmbH · 700037-05/12 · Sous réserve de modifications dans le design et la technique · Printed in Germany