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Instruction Manual

Manual de instrucciones

Mode d'emploi

ECO – MATIC®

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with bottle attachment

con adaptador para frascos

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by **HSW**

for veterinary use only

0,3 ml, 1 ml, 2 ml or 5 ml

Henke-Sass, Wolf GmbH
Keltenstrasse 1
D-78532 Tuttlingen
Germany
Tel. +49 74 62 94 66 166
Fax +49 74 62 94 66 52 08
www.henkesasswolf.de
E-mail: HSW@henkesasswolf.de

ECO-MATIC®
von HSW

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
ECO-MATIC®
by HSW

- Automatic self-filling syringe for injections
- with bottle attachment

- A) ergonomic designed handle
- B) plastic barrel with bottle attachment
- C) durable plastic basket
- D) metal needle fitting, Luer Lock
- E) dosage setting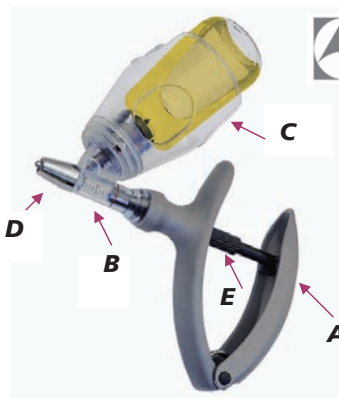

Instructions for use

The ECO-MATIC® self-filling dosing syringe is designed for continuous use, only the O-ring and the barrel can be changed if necessary.

It is essential that the syringe is cleaned before each use. Please follow the manufacturers recommendations with any cleaning or disinfectant solutions used.

1) TO FIT THE NEEDLE

Attention:

For your safety: Leave the protective cover (only on disposable needles) on the needle until ready for use.

- Insert needle onto Luer Lock fitting and secure the needle clockwise.
- Disconnect the needle fitting the opposite way.

2) CONNECT THE BOTTLE

1. Pierce the rubber cap of the bottle with a sterile needle before you attach the bottle into the spike .
2. Insert the bottle into the basket.
3. Line up the bottle, by using the viewing window of the basket, in the centre of the rubber cap and pierce the spike into the bottle while rotating the bottle slightly.

3) EXCHANGE THE BOTTLE


- Pull out the bottle by using the grip hollow on both sides of the bottle.

4) SET DOSE

- For 1 ml, 2ml and 5 ml: Turn the dose adjuster either clockwise or counter – clockwise until front of the piston is aligned with the dose setting printed on the barrel for 0,3 ml.
- For 0,3 ml: Turn the dose adjuster either clockwise or counter – until the front of the O-ring is aligned with the dose setting printed on the barrel.


Dose setting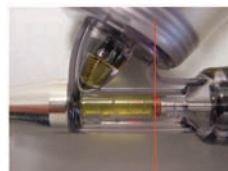

Dose settings

5) FILLING OF THE ECO-MATIC®

1. Remove protection cover of the needle.
2. Hold the ECO-MATIC® at a slight incline and take care that the spike is below the liquid to avoid suck in of air into the barrel.
3. Squeeze adjustable handle and then release quickly to draw fluid into the barrel.
4. To remove air bubble inside the barrel, repeat step (c).
5. To fill the syringe and prepare for injection please make sure that the syringe is adjusted on volume, 0,3 ml (for 0,3 ml syringe) or 0,5 ml (for 1ml syringe) to prime and that no air bubbles are visible. Hold the syringe upright and squeeze the plunger so that air bubbles can be released. From now on any dosage will be possible.

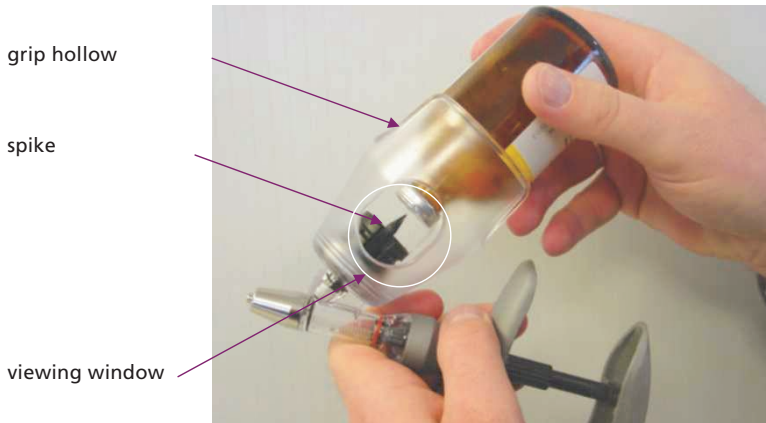INSTRUCTION AFTER USE OF THE ECO-MATIC®

Remove all traces of material from syringe by flushing thoroughly with a hot water detergent mix. Rinse thoroughly with clean, warm water.

DISINFECTION

It is recommended that the ECO-MATIC® and a supply of needles or nozzles be disinfected before and after each use.

1. Suck "hot" water into the barrel by using an attached bottle with warm water.
2. Suspend the complete ECO-MATIC® including needles or nozzles in a container of water and boil for up to 20 minutes.
3. Remove the ECO-MATIC® from container, wrap cloth around the handle and pump it dry.

TO LUBRICATE:

Remove plastic barrel from the handle by turning it. Lubricate piston and glass barrel with 100% pure oil, cold squeezed or equivalent vegetable oil.

Please follow the manufacturers durability and storage life instruction.

Do not use mineral oil, silicone, WD-40, glycerine or alcohol. Petroleum based products harden the piston, reducing the life of the component.

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© copyright · Henke-Sass, Wolf GmbH · 700038-06/08 · Änderungen in Technik und Design vorbehalten · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-06/08 · Subject to amendments in technology and design · Printed in Germany
© copyright · Henke-Sass, Wolf GmbH · 700038-06/08 · Reservados los derechos de modificaciones técnicas y de diseño · Impreso en Alemania
© copyright · Henke-Sass, Wolf GmbH · 700038-06/08 · Sous réserve de modifications dans le design et la technique · Printed in Germany