The Stericap - the biggest breakthrough in injection delivery since the sterile needle

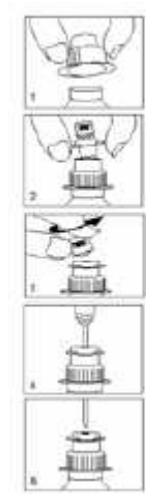
The Stericap is a unique precision moulded container enclosing a plug of foam, which carries a liquid disinfectant. As the needle passes through the Stericap, whilst injecting as normal, it is swabbed clean by the foam and coated with disinfectant that destroys bacterial and viral pathogens.

Single dose system

If a dirty needle is used to withdraw from a bottle it may:

- a) Contaminate bottle contents
- b) Cause infection and abscesses at the injection site
- c) Transmit disease between animals

To reduce these risks the Sterimatic Bottle protector can be used

Here's how it works:

- 1) Remove the flip-off top from the vial. Push vial adaptor over neck of vial.
- 2) Fit Stericap firmly into vial adapter.
- 3) Twist of the white leaf on Stericap.
- 4) The vial is ready for use. Insert needle through Stericap, into bottle and draw off normally. *As the needle passes through the Stericap it is cleaned reducing bottle contamination and also reducing infection and abscessing.*

Each Stericap can be used for up to 100 automatically disinfected, hygienic injection deliveries or within 3 days of opening (for poultry up to 500 injections).

The Stericap has been tested extensively in both laboratory and field conditions where it has proved effective against both bacterial and viral contaminants. Tests show no effect on some live vaccines, this is because the Stericap cleans the outside of the needle and the injectable 'flushing

action' cleans the inside.

The Stericap system has been used around the world by Government departments, major pharmaceutical companies, veterinary practices and farmers for many years. Do not use where disinfectants are contra-indicated. Recommended use in accordance with Sterimatic data-file. Stericap contains 2.5% Gluteraldehyde & 5% Bardac 22.

Helps protect
all livestock against
risk of costly and
dangerous viral and
bacterial cross
infection

Safer delivery
from bottles
Helps protect
bottle contents
Effective against tested
bacterial and viral
pathogens
More hygienic
injections

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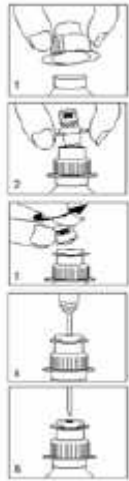
Helps protect all livestock against risk of costly and dangerous viral and bacterial cross infection

**Bottle protector
Single dose systeem:**

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**Safer delivery from bottles
Helps protect bottle contents
Effective against tested bacterial and viral pathogens
More hygienic injections**

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