


# The Ultimate BMV

## OPERATING INSTRUCTIONS

1. Remove the bottle retaining cap from the applicator.
2. Slide the retaining cap onto the product bottle.
3. Gently pierce the rubber cap on the product bottle with the bottle spike and tighten the retaining cap firmly onto the applicator.
4. Set applicator to the maximum dose. To adjust, squeeze the handles together and then turn the dose selector.
5. To prime, hold the applicator with the needle cap pointing vertically upward. Squeeze the handles and then release to allow the barrel to fill with product. Repeat until all air is expelled from the barrel.
6. Rotate dose selector to the desired dose rate.

## BOTTLE CRADLE FITTING INSTRUCTIONS (Optional)

1. Remove plastic safety guard and bottle cap.
2. Screw bottle cradle onto applicator.
3. Connect the product bottle onto the bottle spike by gently piercing the rubber cap.
4. Push on firmly until bottle cap engages with bottle cradle claws.

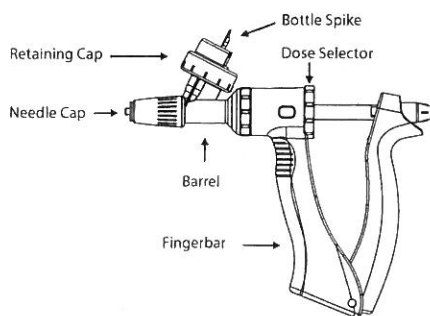


## NEEDLE FITTING INSTRUCTIONS

**Note:** The needle cap does not need to be removed when fitting needles.

1. To attach needle, turn needle into the needle cap in a clockwise direction until secure (finger tight).
2. To remove needle, turn needle in an anti-clockwise direction and pull gently.


## CLEANING INSTRUCTIONS

1. Flush product residue from the barrel.
2. Flush hot, soapy water through the applicator until product is removed. Then flush with clean water.
3. Unscrew the barrel from the fingerbar. Soak the o-ring on the piston with vegetable oil. (**DO NOT USE MINERAL OIL**)
4. Re-attach barrel to the fingerbar and store in a cool, dry place.

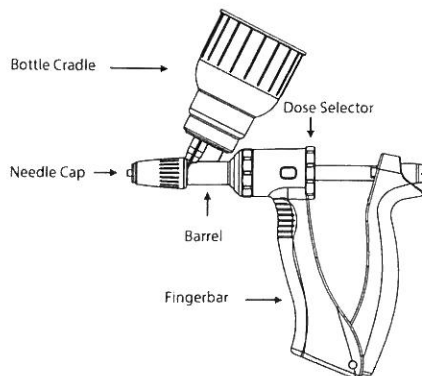

## The Ultimate BMV INSTRUCTION LEAFLET

Please read before use

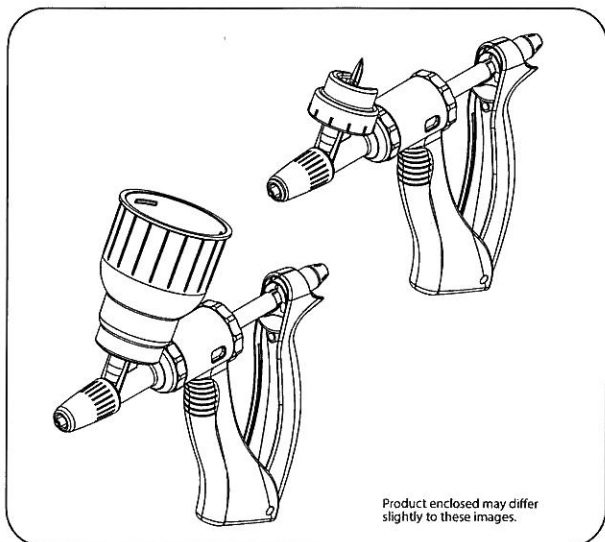

## WARNINGS:

- This applicator is designed to inject animals. It should not be used for any other purpose.
- The operator should seek professional advice on the correct technique before using this applicator.
- The operator must take every precaution to avoid injury or contamination when using this applicator.
- Before use, ensure that the applicator is in good working order. Do not use it if a fault becomes apparent.
- To assist with proper use, the operator must follow the directions of this instruction leaflet.
- To ensure continued operation of the applicator, it must be fully cleaned at the conclusion of each working day.
- The applicator may cease to work correctly if it is not maintained according to the directions of this instruction leaflet.
- It is important that the operator reads and follows the directions on the label of the product about to be administered for use and safety instructions.