
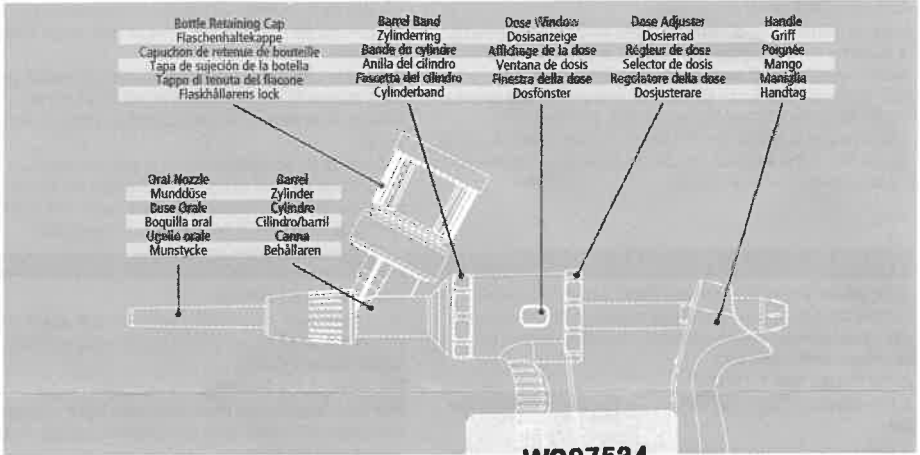
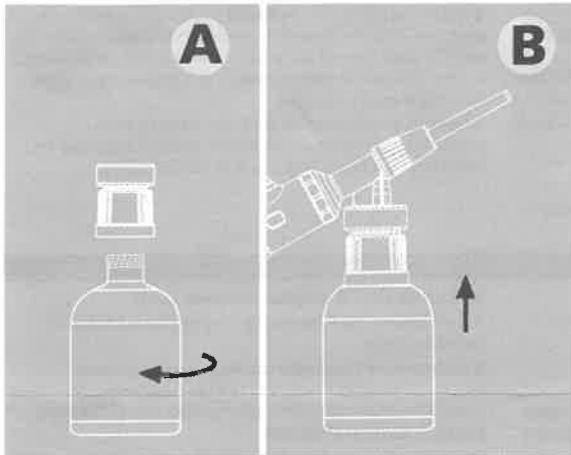
ULTIMATE BMV™ PIGLET DRENCHER

Simcro™ DATAMARS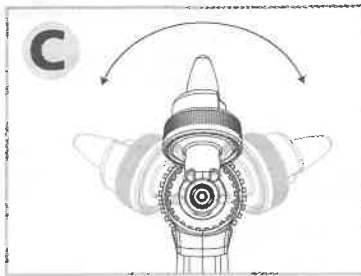

W097634

350 0002-490 | 650846 | RB

ENGLISH

IMPORTANT Always read the label: Check the label on the pharmaceutical manufacturer's container for dose rates, precautions, and safety information prior to use.

INSTRUCTIONS FOR USE

- 1 Screw the retaining cap onto the product bottle (A).
- 2 Remove the plastic safety tube from the bottle spike.
- 3 Tighten the retaining cap firmly onto the applicator (B).
- 4 To adjust dose, squeeze the handles together and then turn the dose adjuster.

Note: The Barrel can be rotated left or right, to suit operator preference to ensure the fluid level of the Product Container is always above the Spike. To rotate the Barrel, first unscrew (anticlockwise) the Barrel Band. Rotate Barrel to desired angle and retighten Barrel Band (C).

To fit nozzle

If the nozzle is not attached, screw nozzle on to applicator, position at required angle and tighten the nut firmly.

To set dose

Turn the dose adjuster to the required dose, as marked on the dose window (selectable). The system is now ready to use.

To prime

- 1 Hold the device pointing upwards at an approximately 45° angle making sure the fluid level in the bottle is always above the inlet spike to avoid air being drawn into the device.
- 2 Pump handle once and release to draw vaccine into the barrel. Repeat if required to expel all air prior to commencing.

Warning: Care must be taken to ensure the liquid does NOT come into contact with any part of the operator's body. Chemicals may cause injury to the operator.

If the unit fails to refill during use

- 1 Check the bottle/pack is attached correctly.
- 2 Disconnect the bottle/pack and flush potential contaminants from valves by submerging injector in warm water and pumping vigorously.

CARE AND MAINTENANCE

To ensure continued high performance from this instrument, attention to cleanliness is essential.

- 1 After each use, flush the instrument and feed tube thoroughly by pumping through a warm water detergent mix, followed by clean water.
- 2 Unscrew barrel and lubricate the Piston O-ring with olive oil or any food grade vegetable oil. Re-fit the barrel and operate applicator to distribute lubricant internally. **DO NOT use Mineral Oil.**
- 3 Store the applicator in a cool, dry place, preferably in a plastic bag to keep it dust free for re-use.

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