

FLUX



The long rumen line.

scientifically
tested

What is **FLUX**?

The long rumen line.

What is the **FLUX** for?

Safe administration of liquids into the rumen.

Why is the **FLUX** safe?

If the mark on the line meets the incisors then the line's head is located the rumen. No regurgitation.

How to use?


Pull the cow's head forward. Head, neck and spinal column shall form a straight line. Insert the line's head into the oral cavity at the tooth free rim of the jaw. Cautiously shove the line down the esophagus into the rumen until the mark on the line meets the incisors. Connect funnel. Administer fluids. Remove **FLUX** cautiously.

How to clean?

Connect **FLUX** to a water tap and rinse it. Lift the line's head up such that the water runs down along the outer probe.

What is the **FLUX** made of?

Stainless steel. With mark. With funnel and GARDENA®-quick coupling connection.

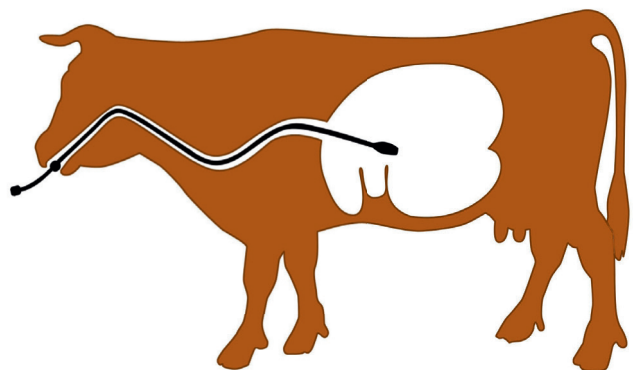

FLUX. Scientifically tested.

Geishauser et al. 2010: Tierärztl. Umsch. 65, 429–434.

FLUX. Safe administration of liquids into the rumen.