MS Mobile drencher

1508250

NL	MS Mobile drencher manual, 45 liter	
	Gebruiksaanwijzing	3
EN	MS Mobile drencher manual, 45 litre	
	Operations manual	4
DE	MS Mobiler Drencher, 45 liter	
	Gebrauchsanweisung.....	5
FR	MS Pompe à drencher manuelle, 45 L	
	Mode d'emploi	6
RU	MS Drenching Mobile Set	
	Инструкция по применению.....	7

MS Mobile drencher manual, 45 liter

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MS Mobile drencher manual, 45 litre

Important moves to successful drenching using MS Mobile Drencher:

1) Prepare MS Mobile Drencher:

- a) Open the Drenching Mobile Set lid.
- b) Mix the drench powder thoroughly with warm water ($\approx 40^{\circ}\text{C}$).
- c) Close the Drenching Mobile Set lid to prevent spillage.

2) Prepare the cow:

- a) Ensure the cow is restrained.
- b) Keep the cow's head low to ensure a direct path to the rumen.
- c) Open the nose clip and insert frick tube carefully.
- d) Engage nose clip to hold tube in place.

3) Insert hose correctly:

- a) Carefully insert hose until you feel resistance in the back of the cow's mouth. Pause and let the cow swallow. DO NOT FORCE. Once the hose has been swallowed, watch the moving hose on the left side of the throat or feel the throat and make sure there are two hard structures. One will be the hose in the esophagus, the other will be the cow's trachea.
- b) Push the hose in all the way until you are in the rumen.
- c) Engage the hose to the frick tube.

4) Listen and smell:

- a) Before attaching the end of the hose to the Drenching Mobile Set pump, hold it to your ear and listen. You should hear a bubbling or gurgling sound. You should not hear breathing.
- b) Smell the end of the hose. You should detect the fermenting odor of the rumen.

5) Pump correctly:

- a) Connect the hose to the Drenching Mobile Set pump and begin pumping. A slow, easy, steady stroke should be used.

6) Drain and remove hose:

- a) Disconnect hose from pump and hold high for 10 seconds so that any remaining fluid drains out.
- b) Blow any remaining fluid from the hose.
- c) Disconnect hose from the frick tube and remove it carefully until you feel resistance. Then disconnect nose clip and remove the frick tube.

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**Schippers Italia Srl**
**Schippers Schweiz GmbH**
**Schippers UK Ltd.**
**LLC Schippers Russia**