

JUNIOR[®]



The first ergonomic pill applicator for calves.

scientifically tested

What is the JUNIOR?

The first ergonomic pill applicator for calves. Handier and gentler.

What is the JUNIOR for?

Suckling calves.

Why is it ergonomic?

The JUNIOR is curved like the route through the oral cavity to the

esophagus. The cup of the JUNIOR is rounded.

Why is it handier?

The JUNIOR lies comfortably in the hand. No grip change.

How to use?

Insert a pill into the cup. Grab the tooth-free rim of the palate.

Tickle the palate until the calf opens the mouth itself. Carefully insert the JUNIOR into the oral cavity. Slide it carefully across the base of the tongue. Release the pill.

How to clean?

Rinse with hot water.



Tickle the palate until the mouth opens.



Carefully insert the applicator.



JUNIOR. Ergonomic.

JUNIOR. Scientifically tested.

JUNIOR[®]. Handier and gentler.