

SENIOR[®]



The first ergonomic pill applicator for cows.

scientifically
tested

What is the **SENIOR**?

The first ergonomic pill applicator for cows. Handier and gentler.

What is the **SENIOR** for?

Dairy cows.

Why is it ergonomic?

The **SENIOR** is curved like the route through the oral cavity to the esophagus. Its end is rounded.

Why is it handier?

The **SENIOR** lies comfortably in the hand.

What is the **SENIOR** made of?

Tube of stainless steel, plunger of plastic.

How to use?

Insert a pill. Grab the tooth-free rim of the palate. Tickle the palate until the cow opens the mouth herself.

Carefully insert the **SENIOR** into the oral cavity. Slide it carefully across the base of the tongue until the red mark meets the muzzle. Release the pill.

How to clean?

Rinse with running hot water.



Tickle the palate until the mouth opens.



Carefully insert the applicator.



SENIOR. Ergonomic.

SENIOR. Scientifically tested.

SENIOR[®]. Handier and gentler.