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(EN) Hygiene in and around needleless treatment equipment.

Pay sufficient attention to hygiene in and around the equipment. Needle free equipment should be treated such as normal syringes. Thorough cleaning after use is very important. Work strictly according to following steps:

Protocol rinse and clean needle free treatment equipment.

1. First clean the equipment on the outside with warm water and detergent. (do not fully submerge).
2. Place a bottle MS Rinse Liquid on the pump unit. Pump as long as necessary until flushing fluid is completely clean.
3. Then place a bottle MS Contact Liquid on the pump unit. Pump until the fluid comes out of the syringe. (depending on dosing tube length, see diagram below)
4. Leave the syringe with fluid like this until the next treatment. Do not remove the bottle MS Contact Liquid from the pump unit. (Tests have shown that MS Contact Liquid has no negative effect on the parts in the syringe)
5. Disinfect all outside parts with P3 alcodes.
6. Before starting with next treatment, place MS Rinse Liquid on the pump unit and pump as long as necessary to remove the MS Contact Liquid from the pump. To ensure all MS Contact Liquid is removed from the system, calculate with the values in the diagram below. Check which dosage plug is fitted.
7. Place the inject fluid on the pump unit, pump as long as necessary to remove the MS Rinse Liquid from the pump. Calculate with the values in the diagram below. Check the pressure setting and the system is ready to use.

Slanglengte/Tube length	Inhoud slang/Content in tube
400 mm	5cc (cm ³)
1600 mm	8cc (cm ³)
3000 mm	12cc (cm ³)

Artikelnummers/Article numbers:	
2509250-12	MS Contact Liquid, 20 ml
2509251-12	MS Rinse Liquid, 20 ml
2509928	P3 alcodes, 5L

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The following parts can be cleaned in an ultrasonic bath.

Do not place complete dosing units and injection guns in an ultrasonic bath.

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