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Instructions for a.s Test kit

(diarrhoea for calves)

The a.s Test kit for the qualitative detection of rota viruses, bovine corona viruses (BCV), E-coli and Cryptosporidium parvum in the faeces of calves.

- 1 STake an adequate stool sample from the calf or calves whose faeces you wish to test.

- Note
- Ensure that sufficient stool is picked up to allow a good, 'clean' sample to be taken.
 - If possible: avoid picking up mucus when taking the sample.
 - Ensure that there is only contact with faeces when taking the sample. Contact with the stall floor or straw can falsify the results.


- 2 Using the pipette, extract faeces from the stool sample.

- Note
- If the faeces are too thick to be extracted using the pipette, place a small amount on the applicator. This can be found in the so-called 'sample container' – the transparent bottle with a blue screw cap. This blue screw cap has a white stick with a channel in which to place faeces (the applicator). Ensure there are sufficient faeces in this channel, so that the test fluid in the bottle is clearly discoloured once the cap is replaced.

- 3 Add 4 to 5 drops of faeces to the sample bottle containing the fluid (bottle with the blue screw cap). If there are sufficient faeces in the bottle, the liquid will become discoloured.

- 4 Close the sample bottle using the blue cap and shake it lightly.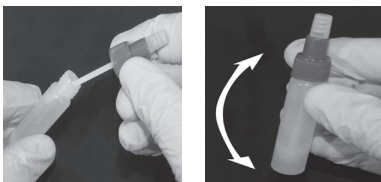

- 5 Remove the cassette from the white aluminium packaging and place it on a level surface.
- 6 Unscrew the white top from the bottle with the blue screw cap and drop 4 to 5 drops of the stool sample into the sample wells.


- Note
- If nothing happens within a few minutes (there is no reaction in the four display panels), add some extra drops of the stool sample into the sample wells.
 - Make sure the stool sample does not drip onto the four display panels.


- 7 Wait for around 10 minutes.

- 8 The markings in the display panels show whether the test is positive or negative for Rota viruses, Corona viruses, E-coli and/or Cryptosporidium. A positive test result is shown when the T and C markings are both visible. This means that the calf is infected with Rota virus and Cryptosporidium, in this example.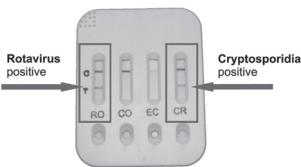

A negative test result is shown when only the C marking is visible. This means that the diarrhoea is not caused by Corona virus and E-coli, in this example.

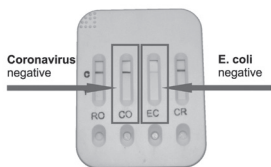Storage and use

The a.s Test kit must be stored and carried out in a dry location and at a temperature between 2 °C and 30 °C.

Liability

The complete liability risks in relation to the use of this product are borne by the purchaser. The manufacturer assumes no liability for indirect, special or consequential damage of any kind whatsoever which results from the use of, the carrying out and evaluation of tests with this product.

Contents of a.s Test kit

- 5 x Test cassettes in white aluminium packaging
- 5 x Sample containers with test fluid
- 5 x Pipettes
- 1 x Description and instructions

Test for single use. After opening the white aluminium packaging containing the test cassette, carry out the test as soon as possible. The test cassette packaging must not be damaged prior to use. During testing, observe the prescribed 10-minute evaluation time.

Sensitivity and specificity of the a.s test kit

	Sensitivity ¹	Spezifität ²
Rota viruses	99.0%	98.0%
Bovine corona viruses	98.4%	98.0%
E-coli-K99	97.8%	99.0%
Cryptosporidium parvum	98.2%	99.0%

Study vs. ELISA

- ¹ Sensitivity: the number of correct positive tests, i.e. the animal was infected with the virus/bacteria in question
- ² Specificity: the number of correct negative tests, i.e. the animal was not infected with the virus/bacteria in question

In case of digestive disorders (diarrhoea) it is recommended to use Topro DiarFix. Topro DiarFix is preferably administered by the milk to stabilise the water and electrolyte balance. Topro DiarFix also supports recovery during digestive disorders and provides the necessary intestinal protection.

a.s. is a registered trademark off Bio Enterprise B.V.
www.bio-enterprise.nl

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a. s ist ein eingetragenes Warenzeichen von Bio Enterprise B.V.