
MS Hands-free Oral Dispenser 3707500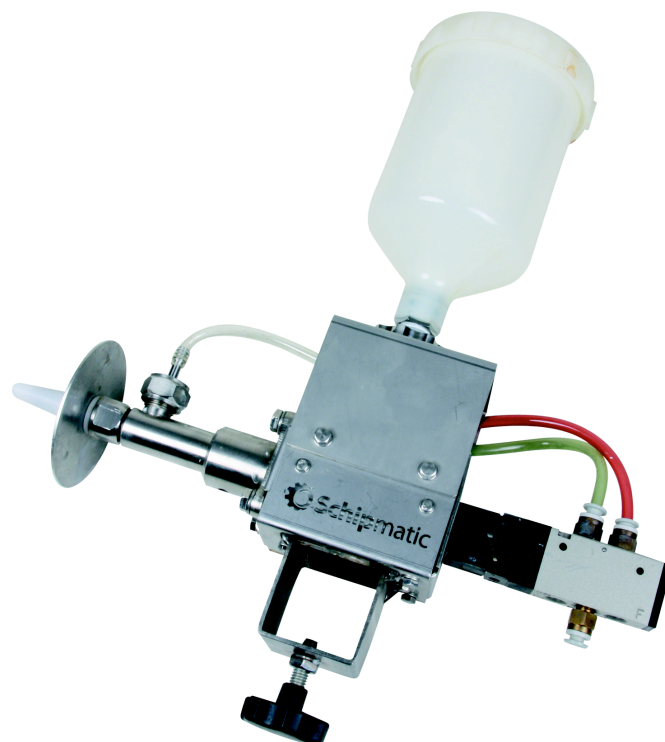

NL	MS Hands-free oral dispenser Gebruiksaanwijzing	4
EN	MS Hands-free oral dispenser Manual.....	6
DE	MS Hands-free oral dispenser Gebrauchsanweisung	8
FR	MS Hands-free oral dispenser Mode d'emploi.	10
ES	MS Hands-free oral dispenser Instrucciones para el uso.....	12
IT	MS Hands-free oral dispenser Manuale di istruzioni.....	14

Technische gegevens:

Werkt volledig pneumatisch

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Werkdruk: 5-6 bar

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Technical data:

- Full pneumatic action
- Reservoir capacity: 600cc (to treat 850 piglets with a standard dose of 0.7cc/shot)
- Dose: 0.7cc
- Connection: 6mm air hose
- Operating pressure: 5-6 bar
- Number of doses in the system: 6 (x 0.7cc)

Installation of the MS Oral Dispenser

The MS Oral Dispenser can be easily installed on the existing Multi treatment unit. As the needle-free syringe(s), it can be mounted on the T-support of the treatment unit (photo 1). If there is not enough space on the T-support for the Oral Dispenser, a longer T-support is also available (extended T-support, product no. 3408923).

The MS Oral Dispenser is supplied with a length of air hose and a T-connector which can be used to easily connect the Oral Dispenser to the existing air system (photo 2a and 2b).

Use of the MS Oral Dispenser

Filling and operating the MS Oral Dispenser

The reservoir (maximum capacity 600cc) on top of the Oral Dispenser can be filled with the fluid to be administered to the piglets (photo 3). If the Oral Dispenser has not contained fluid previously (e.g. if this is its first use), any air present should first be removed. This is best done by filling the reservoir with water. Activating the Oral Dispenser several times in quick succession will expel any air from the system. The Oral Dispenser is activated by pressing on the push disk, which has around a centimetre of free movement before touching the cylinder (photo 4). Activate the Oral Dispenser manually until a constant volume of fluid emerges from the nozzle with each activation and no more air bubbles can be seen in the transparent hose. Any water remaining in the reservoir can be discarded and the treatment fluid poured in. Activating the Oral Dispenser a further six times will remove the water from the system and cue up the treatment fluid. The Oral Dispenser is now ready for use.

Administering fluids to piglets

The best way to treat a piglet using the Oral Dispenser is to start by gripping the piglet firmly in both hands. Place one hand under the belly and use the other to hold the piglet's mouth open. Then move the piglet straight ahead with its snout against the push disk so that the fluid is ejected directly into the back of its throat. If the piglet is held at an angle, some or all of the fluid may not enter the piglet's mouth correctly and fluid may be lost.



Maintenance on the MS Oral Dispenser

Flushing and disinfecting the MS Oral Dispenser

To guarantee correct dosage and proper functioning of the Oral Dispenser, the system must be flushed after each use.

Fluid remaining in the reservoir after treating piglets can be easily poured back into the original product bottle to avoid wastage (photo 5).

Rinse the reservoir with water. Then flush the system (using clean water in the reservoir) by activating the Oral Dispenser manually several times. As an additional disinfection measure, flush again using MS Surdes (product no. 2509255 - MS Surdes, 1L). After flushing, check the filter in the non-return valve on the suction side of the Oral Dispenser for visible dirt or blockages (photo 6a and 6b).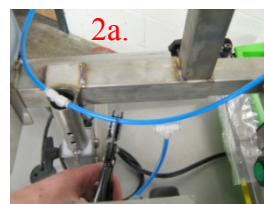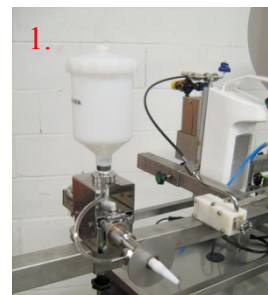


Seal maintenance

Several parts of the Oral Dispenser are sealed using an o-ring. The picture below shows the locations of the different o-rings (photo 7).

Most of these are static seals and do not need to be replaced periodically.


Only o-ring G on the piston of the Oral Dispenser needs to be replaced after every 12,000 treatments.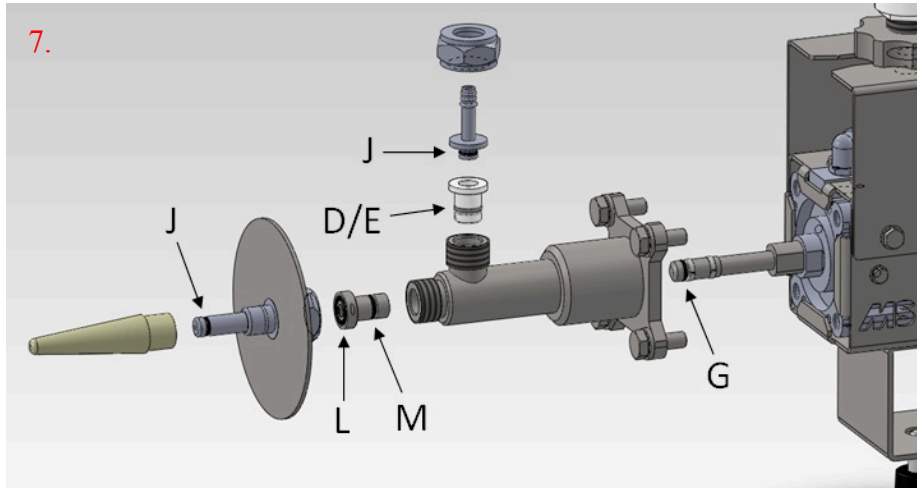

To replace the o-ring on the piston, the piston housing of the Oral Dispenser must be disassembled. It is recommended that this is done after the system has been flushed and the reservoir emptied of any rinsing fluid.

The piston housing is mounted on the air cylinder by means of four bolts. Once these have been unscrewed, the piston housing complete with reservoir can be removed from the Oral Dispenser (photo 8a, 8b and 8c). The o-ring can now be replaced by pulling it off the piston with a pair of tweezers and fitting the new o-ring in its place. Apply a little silicone spray to allow the piston housing to fit back over the piston easily and to extend the life of the o-ring (photo 9a, 9b and 9c).


Mount the piston housing and reservoir exactly as they were before. It is important to ensure that the hose connection on the piston housing points to the left or right. If this is not done, the non-return valve inside may not seal properly and the Oral Dispenser will not give the correct dosage.

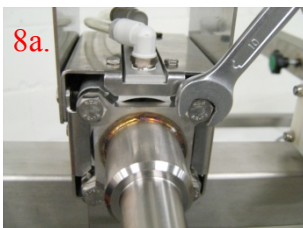Unscrew the four piston housing bolts.

Remove the piston housing complete with reservoir from the Oral Dispenser.

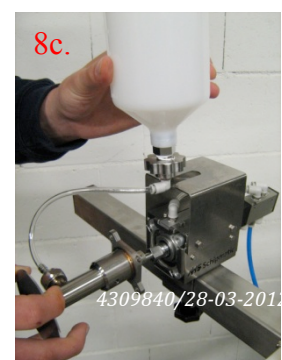← (left)
Remove the old (worn) o-ring.
(right) →
Fit the new o-ring over the piston, in front of the white ring.

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**Schippers Italia Srl**
**Schippers Schweiz GmbH**
**Schippers UK Ltd.**
**LLC Schippers Russia**
**Schippers Canada Ltd**
7102 52nd Street Bay # 18 • Lacombe, AB • T4L 1Y9