


EN

NL

FR




PT



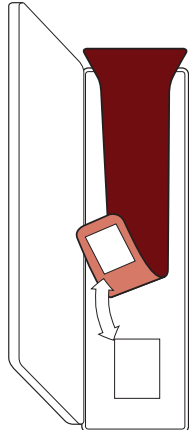

1.

- EN** Place the SmartSleeve™ inside the IFT SmartWarmer™
- NL** Plaats de SmartSleeve™ in de IFT SmartWarmer™
- FR** Placez le SmartSleeve™ à l'intérieur de l'IFT SmartWarmer™
- PT** Coloque o SmartSleeve™ dentro do IFT SmartWarmer™

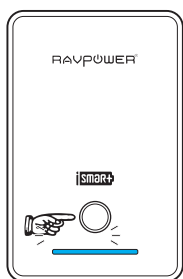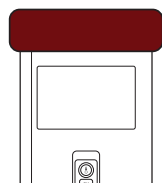
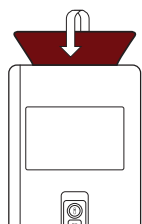


2.

- EN** Check the power bank battery level
- NL** Controleer het batterij niveau
- FR** Vérifier le niveau de batterie de la batterie d'alimentation
- PT** Verifique o nível da bateria do banco de alimentação

3.

- EN** Insert the portable power bank inside the pocket
- NL** Plaats de powerbank in de zak
- FR** Insérez la batterie portable dans la poche
- PT** Insira o banco de energia portátil dentro do bolso

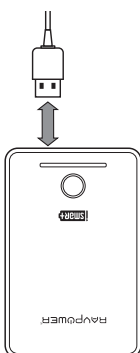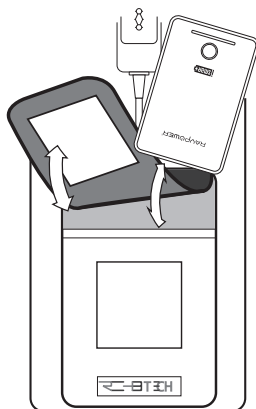


4.

- EN** Plug USB cable into the portable power bank
- NL** Verbind de USB met de powerbank
- FR** Branchez le câble USB dans la batterie portable
- PT** Conecte o cabo USB no banco de energia portátil

5.

- EN** Press the button to select the temperature
- NL** Druk op de knop om temperatuur te selecteren
- FR** Appuyez sur le bouton pour changer la température
- PT** Pressione o botão para mudar a temperatura

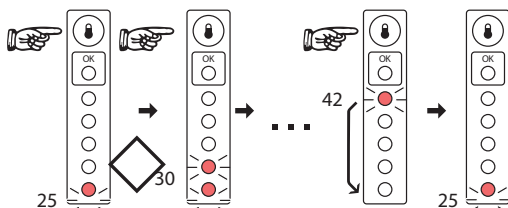


6.

- EN** When the green light is on, the device is ready to use
- NL** Bij groen licht is het apparaat gereed voor gebruik
- FR** Lorsque allumez le feu vert est prêt à l'emploi
- PT** Quando acender a luz verde está pronto para uso





- When the temperature is above the set value, all red lights will turn on.
- Als de temperatuur boven de ingestelde waarde is branden alle lampjes rood.
- Lorsque la température est au-dessus de la valeur réglée, tous les feux rouges seront allumés
- Quando a temperatura está acima do valor definido, todas as luzes vermelhas se acenderá




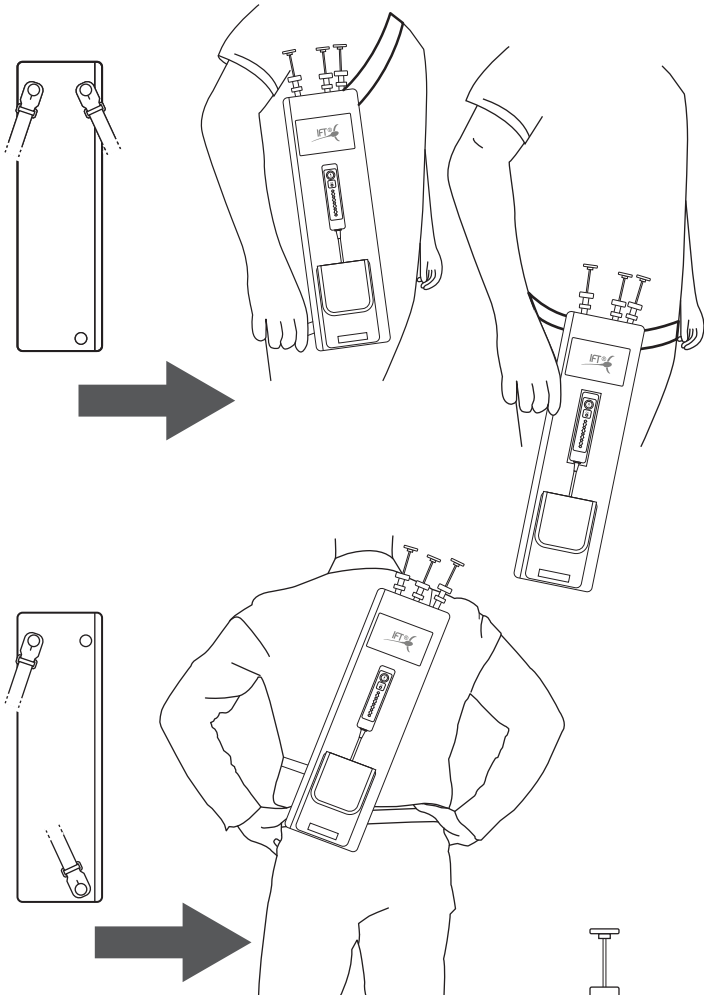


- EN** Place the AI gun until it stops at the bottom
- NL** Plaats de pistolet helemaal totaan de bodem
- FR** Mettez la pointe du pistolet d'insémination jusqu'à ce qu'il touche le fond
- PT** Coloque o aplicador até parar no fundo

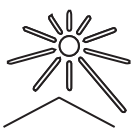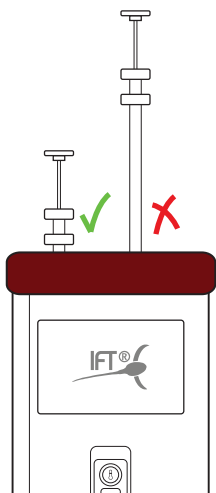


- EN** Do not store in direct sunlight
- NL** Niet blootstellen aan direct zonlicht
- FR** Ne pas stocker à la lumière directe du soleil
- PT** Não armazene sob luz solar direta

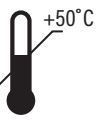

- EN** Do not apply major bending force to the product
- NL** Het product niet buigen
- FR** N'appliquez pas de force de flexion importante au produit
- PT** Não aplique grande força de flexão ao produto

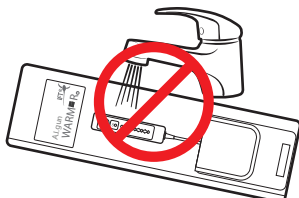

- EN** Do not submerge
- NL** Niet waterbestendig
- FR** Ne pas plonger
- PT** Não submergir o aparelho

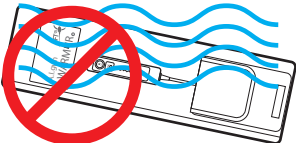

**EN****Precautions:**

Do not apply major bending force to the product as this may damage the electronics  
 Store the product and in a clean and dry environment  
 The product can be cleaned using a damp cloth, moist towelettes or hand sanitizing wipes.

**Maintenance, Care of Portable Power Bank**

Do not over charge your power bank.  
 Handle your power bank carefully just like any other electronic device.  
 Knocking or dropping your power bank may damage the internal electronics which can degrade performance.  
 Use your power bank frequently. At the very minimum, fully discharge and recharge the power bank once a month. This will help maintain battery and increase the lifespan.  
 Do not try to disassemble your power bank. This may lead to consequences such as overheating and short circuit and will void warranty.  
 Avoid exposing the power bank to extreme heat. Overheating and excessive moisture may damage the power bank.

**NL****Voorzorgsmaatregelen:**

Probeer het product niet te buigen, dit kan de elektronica beschadigen.  
 Het product bewaren in een schone en droge omgeving.  
 Het product kan schoongemaakt worden met een vochtige doek of handreinigingsdoekjes

**Onderhoud van de powerbank**

De powerbank niet overladen.  
 Behandel de powerbank voorzichtig net als andere elektronische apparaten.  
 Stoten of laten vallen van de powerbank kan schade veroorzaken aan de elektronica.  
 Gebruik de powerbank regelmatig. Minimaal één keer per maand volledig opladen en ontladen.  
 Haal de powerbank niet uit elkaar, dit kan oververhitting of kortsluiting veroorzaken en uw garantie komt te vervallen. Niet blootstellen aan extreme hitte.

**FR****Précautions:**

N'appliquez pas de force de flexion importante au produit car cela pourrait endommager l'électronique  
 Stocker le produit dans un environnement propre et sec  
 Le produit peut être nettoyé à l'aide d'un chiffon humide, de serviettes humides ou de lingettes désinfectantes pour les mains.

**Maintien, entretien de la banque d'énergie portable**

Ne chargez pas trop votre banque d'alimentation.  
 Manipulez votre banque d'alimentation avec soin tout comme n'importe quel autre appareil électronique.  
 Le fait de heurter ou de laisser tomber votre banque d'alimentation peut endommager l'électronique interne qui peut dégrader les performances.  
 Utilisez fréquemment votre banque d'alimentation. Au minimum, déchargez complètement et rechargez la banque de puissance une fois par mois. Cela aidera à maintenir la batterie et augmenter la durée de vie.  
 N'essayez pas de démonter votre banque d'alimentation. Cela peut entraîner des conséquences telles que la surchauffe et les courts-circuits et annule la garantie.  
 Évitez d'exposer la batterie à une chaleur extrême. Une surchauffe et une humidité excessive peuvent endommager la batterie.

**PT****Precauções:**

Não aplique grande força de flexão ao produto, pois isso pode danificar a eletrônica  
 Armazenar o produto em ambiente limpo e seco  
 O produto pode ser limpo com um pano úmido, toalhas úmidas ou toalhetes sanitizantes para as mãos.

**Specifications:**

Temperature accuracy: +/- 0.5°C  
 Power: 10 W  
 Size: 47cm x 16cm x 7cm  
 Weight: 0.650kg Including Power Bank

**Power Bank recommendations:**

Output: 5V, 2A  
 Nominal capacity: 10,400mAh (37Wh)  
 Charging time: 6hrs @ 2A  
 Operating time: 5°C ambient - set @ 36°C - 7hrs continuous.

## Warranty And Disclaimer

- 1) This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, IFT Instruments S.R.L. will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.
- 2) Repair or replacement done by IFT Instruments S.R.L. or any of its authorised dealers or repair centres, will repair or replace the Product at no charge, using new or refurbished replacement parts.
- 3) This limited warranty is void if the product's failure has resulted from accident, abuse, misapplication, modification, use of an improper hardware, or unauthorised maintenance or repair. Hazards such as lightning, flood, exceeding voltage specifications, or customer abuse are not covered under warranty.
- 4) The customer must obtain a return material authorization (RMA) number from IFT Instruments S.R.L. before returning any products to IFT Instruments S.R.L. for repair. An RMA number must be issued whether the repair is covered by repair program or not. Customers can obtain RMA numbers by contacting IFT Instruments S.R.L. or any of its authorised distributors.
- 5) All repairs carry a 90-day warranty, which begins the day the repaired item is shipped back to the customer. The product will continue to be covered by any existing repair program or by the 90-day repair warranty, whichever is longer.
- 6) The customer is notified if, after examining and testing a returned product, IFT Instruments SRL concludes that the product is not defective. The product is returned at the customer's expense and the customer is charged a nominal examination and testing fee or the standard repair fee, whichever is less.
- 7) The period of this LIMITED WARRANTY IS 2 YEARS since the retail purchase of the product. To be able to claim any repair, the buyer must supply the retail purchase invoice, with the serial number corresponding to the device.
- 8) In no event shall IFT Instruments S.R.L. be liable for any direct, indirect, punitive, incidental, special consequential damages, to property or life, whatsoever arising out of or connected with the use or misuse of its products.
- 9) This warranty and disclaimer statement are governed by the laws of Uruguay. You hereby consent to the exclusive jurisdiction and venue of the Courts of Uruguay, in all disputes arising out of or relating to the use of this product. Use of this product is unauthorised in any jurisdiction that does not give effect to all provisions of these terms and conditions, including without limitation this paragraph.

## Warranty And Disclaimer

- 1) This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, IFT Instruments S.R.L. will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.
- 2) Repair or replacement done by IFT Instruments S.R.L. or any of its authorised dealers or repair centres, will repair or replace the Product at no charge, using new or refurbished replacement parts.
- 3) This limited warranty is void if the product's failure has resulted from accident, abuse, misapplication, modification, use of an improper hardware, or unauthorised maintenance or repair. Hazards such as lightning, flood, exceeding voltage specifications, or customer abuse are not covered under warranty.
- 4) The customer must obtain a return material authorization (RMA) number from IFT Instruments S.R.L. before returning any products to IFT Instruments S.R.L. for repair. An RMA number must be issued whether the repair is covered by repair program or not. Customers can obtain RMA numbers by contacting IFT Instruments S.R.L. or any of its authorised distributors.
- 5) All repairs carry a 90-day warranty, which begins the day the repaired item is shipped back to the customer. The product will continue to be covered by any existing repair program or by the 90-day repair warranty, whichever is longer.
- 6) The customer is notified if, after examining and testing a returned product, IFT Instruments SRL concludes that the product is not defective. The product is returned at the customer's expense and the customer is charged a nominal examination and testing fee or the standard repair fee, whichever is less.
- 7) The period of this LIMITED WARRANTY IS 2 YEARS since the retail purchase of the product. To be able to claim any repair, the buyer must supply the retail purchase invoice, with the serial number corresponding to the device.
- 8) In no event shall IFT Instruments S.R.L. be liable for any direct, indirect, punitive, incidental, special consequential damages, to property or life, whatsoever arising out of or connected with the use or misuse of its products.
- 9) This warranty and disclaimer statement are governed by the laws of Uruguay. You hereby consent to the exclusive jurisdiction and venue of the Courts of Uruguay, in all disputes arising out of or relating to the use of this product. Use of this product is unauthorised in any jurisdiction that does not give effect to all provisions of these terms and conditions, including without limitation this paragraph.

## Warranty And Disclaimer

- 1) This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, IFT Instruments S.R.L. will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.
- 2) Repair or replacement done by IFT Instruments S.R.L. or any of its authorised dealers or repair centres, will repair or replace the Product at no charge, using new or refurbished replacement parts.
- 3) This limited warranty is void if the product's failure has resulted from accident, abuse, misapplication, modification, use of an improper hardware, or unauthorised maintenance or repair. Hazards such as lightning, flood, exceeding voltage specifications, or customer abuse are not covered under warranty.
- 4) The customer must obtain a return material authorization (RMA) number from IFT Instruments S.R.L. before returning any products to IFT Instruments S.R.L. for repair. An RMA number must be issued whether the repair is covered by repair program or not. Customers can obtain RMA numbers by contacting IFT Instruments S.R.L. or any of its authorised distributors.
- 5) All repairs carry a 90-day warranty, which begins the day the repaired item is shipped back to the customer. The product will continue to be covered by any existing repair program or by the 90-day repair warranty, whichever is longer.
- 6) The customer is notified if, after examining and testing a returned product, IFT Instruments SRL concludes that the product is not defective. The product is returned at the customer's expense and the customer is charged a nominal examination and testing fee or the standard repair fee, whichever is less.
- 7) The period of this LIMITED WARRANTY IS 2 YEARS since the retail purchase of the product. To be able to claim any repair, the buyer must supply the retail purchase invoice, with the serial number corresponding to the device.
- 8) In no event shall IFT Instruments S.R.L. be liable for any direct, indirect, punitive, incidental, special consequential damages, to property or life, whatsoever arising out of or connected with the use or misuse of its products.
- 9) This warranty and disclaimer statement are governed by the laws of Uruguay. You hereby consent to the exclusive jurisdiction and venue of the Courts of Uruguay, in all disputes arising out of or relating to the use of this product. Use of this product is unauthorised in any jurisdiction that does not give effect to all provisions of these terms and conditions, including without limitation this paragraph.

## Warranty And Disclaimer

- 1) This Limited Warranty covers any defects in material or workmanship under normal use during the Warranty Period. During the Warranty Period, IFT Instruments S.R.L. will repair or replace, at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance.
- 2) Repair or replacement done by IFT Instruments S.R.L. or any of its authorised dealers or repair centres, will repair or replace the Product at no charge, using new or refurbished replacement parts.
- 3) This limited warranty is void if the product's failure has resulted from accident, abuse, misapplication, modification, use of an improper hardware, or unauthorised maintenance or repair. Hazards such as lightning, flood, exceeding voltage specifications, or customer abuse are not covered under warranty.
- 4) The customer must obtain a return material authorization (RMA) number from IFT Instruments S.R.L. before returning any products to IFT Instruments S.R.L. for repair. An RMA number must be issued whether the repair is covered by repair program or not. Customers can obtain RMA numbers by contacting IFT Instruments S.R.L. or any of its authorised distributors.
- 5) All repairs carry a 90-day warranty, which begins the day the repaired item is shipped back to the customer. The product will continue to be covered by any existing repair program or by the 90-day repair warranty, whichever is longer.
- 6) The customer is notified if, after examining and testing a returned product, IFT Instruments SRL concludes that the product is not defective. The product is returned at the customer's expense and the customer is charged a nominal examination and testing fee or the standard repair fee, whichever is less.
- 7) The period of this LIMITED WARRANTY IS 2 YEARS since the retail purchase of the product. To be able to claim any repair, the buyer must supply the retail purchase invoice, with the serial number corresponding to the device.
- 8) In no event shall IFT Instruments S.R.L. be liable for any direct, indirect, punitive, incidental, special consequential damages, to property or life, whatsoever arising out of or connected with the use or misuse of its products.
- 9) This warranty and disclaimer statement are governed by the laws of Uruguay. You hereby consent to the exclusive jurisdiction and venue of the Courts of Uruguay, in all disputes arising out of or relating to the use of this product. Use of this product is unauthorised in any jurisdiction that does not give effect to all provisions of these terms and conditions, including without limitation this paragraph.