MS Schippers

One-stop-shop for livestock farmers

MS Semenstorage Pro, 150 L

EN Contents	3
NL Inhoud	11
DE Inhaltsübersicht	19
FR Sommaire	27
ES Índice	35

© Copyright 2014

No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm, audio tape, electronically or by any other means without the prior written permission of Schippers Europe b.v.

Contents

Provisions	4
Introduction	5
MS Semen storage	6
Recommendations	6
Installing the MS Semen Storage Pro 150	6
Technical specifications	7
Precautionary measures and safety instructions	8
The MS Semen Storage PRO 150 Display	9
Adjusting the default temperature	10

Provisions

MS Schippers assumes no responsibility for errors contained herein or for any direct or indirect damages or consequential damages of any nature whatsoever arising from or in connection with provision, function or use of this document.

MS Schippers reserves the right at any time, without prior notice, to make changes to the structure of equipment or the information in the manual, without any resulting obligation to apply these changes to products already delivered.

In case of misuse, the manufacturer's guarantee and responsibility becomes void.

MS Semen Storage Pro 150

Introduction

This manual provides instructions on installing, using and maintaining the MS Semen Storage Pro 150 – to ensure correct installation and operation. You should only perform installation after carefully reading and understanding this manual.

Improper use of the installation could cause danger or damage to the installation or other goods and materials. Therefore, only use the installation in technically perfect condition, and only for the purpose for which it was designed.

No technical changes to the installation are permitted, unless the manufacturer has given express written permission. Without the abovementioned permission, the manufacturer's warranty becomes void. This guide contains information regarding operation, maintenance and tracing and troubleshooting faults. You should take note of all relevant information in this manual before installing or operating the system. This is important for averting material damage and personal injury. Careful and precise assembly results in good operation.

In the design of this installation, MS Schippers has given special attention to safety during installation, operation and maintenance. However, it remains the user's responsibility to observe the safety regulations during working operations.

The MS Semen Storage Pro 150 is a sperm storage cabinet with a whopping 150-litre volume, suitable for Professional farmers and breeders. Temperature control is extremely accurate, with additional cooling and heating capacity to maintain the optimum temperature, keeping it stable and consistent.

MS Semen storage

The 150-litre MS Semen Storage Pro 150 is especially designed for storing diluted semen at 17 °C. It has built-in heating, cooling and air circulation, which keeps the internal temperature constant at 17 °C in all ambient temperatures between -10 °C and + 35 °C.

Recommendations

- The temperature inside the box may differ from the display, always check with a separate temperature gauge to prevent damage.
- Keep the MS Semen Storage Pro 150 upright at all times during transport or movement.
- Avoid shocks during movements.
- We advise you to turn off the MS Semen Storage Pro 150 when not in use, to save electricity.
- Do not place the MS Semen Storage Pro 150 in direct sunlight, because that will lead to higher energy consumption.
- Do not install the MS Semen Storage Pro 150 in animal housing or other possibly corrosive environments.
- Before using the MS Semen Storage Pro 150, always check the temperature, shown on the door-mounted thermometer.
- Do not overfill the MS Semen Storage Pro 150. Semen packs should be laid flat on the shelves.
- Do not place any products against the grills at the rear of the MS Semen Storage Pro 150. Proper air circulation is essential.
- The MS Semen Storage Pro 150 comprises no user serviceable parts. In case of problems, always contact customer service at MS Schippers.

Installing the MS Semen Storage Pro 150

- Place the MS Semen Storage Pro 150 on a flat, dry, horizontal surface in a dust-free environment.
- To ensure proper air circulation, you should always allow at least 150 mm free space on either side of the MS Semen Storage Pro 150.
- **Leave the unit switched off during the first 24 hours after transport!**
- Plug into a standard 230 V 13 A mains outlet.
- The MS Semen Storage Pro 150 will take a certain amount of time to reach the set temperature, depending on the ambient temperature.

Technical specifications

Model	MS Semen Storage Pro 150
Capacity	150 L
Cooling agent	R-134 Fluorine-free coolant
Power used	230 V ~ 50 HZ
Fuse	5 x 20 mm, F5AL 250 V
Utilisation	Ambient temperature: 5-35 °C, Humidity: ≤95%
	Air pressure: 860 - 1060 hPa
	For indoor use
Temperature range	0-60 °C
Accuracy	± 0.5 °C
Temperature fluctuation	± 1 °C
Maximum power consumption	500 W
Storage space dimensions (L x W x H) in mm	480 x 500 x 600 mm
Dimensions of the appliance (L x W x H) in mm	700 x 600 x 1050 mm
	790 x 675 x 1175 mm, including packaging
Weight	Gross: 81 kg, Net: 64 kg
Trays	3

Precautionary measures and safety instructions

This appliance complies with safety regulations applicable within the European Union.


We use the following symbols in this manual, to ensure proper and safe use of the MS Semen Storage Pro 150.

	This symbol indicates a tip
	This symbol indicates a general point for attention
	This symbol indicates a situation that requires extra attention to prevent direct contact with parts that may carry a live electrical current
	Hot objects

The MS Semen Storage PRO 150 Display

- (1) "RUN" will illuminate when the MS Semen Storage Pro 150 is running.
- (2) "HEAT" will illuminate when the heating element is active, and will go out when it is switched off.
- (3) "COOL" will illuminate when the storage cabinet is cooling, and will go out when refrigeration is disengaged.
- (4) "ALMI" will illuminate and a buzzer will sound whenever the measured temperature varies from the required level by more than 3 degrees.
- (5) "▲" Up – pressing this button increases the value showing in the display. Holding this button down will increase the value rapidly. When used on a display with no value showing, this button serves to increase display clarity.
When the display light is out you can push this button to put it back on
- (6) "▼" Down – pressing this button decreases the value showing in the display. Holding this button down will decrease the value rapidly.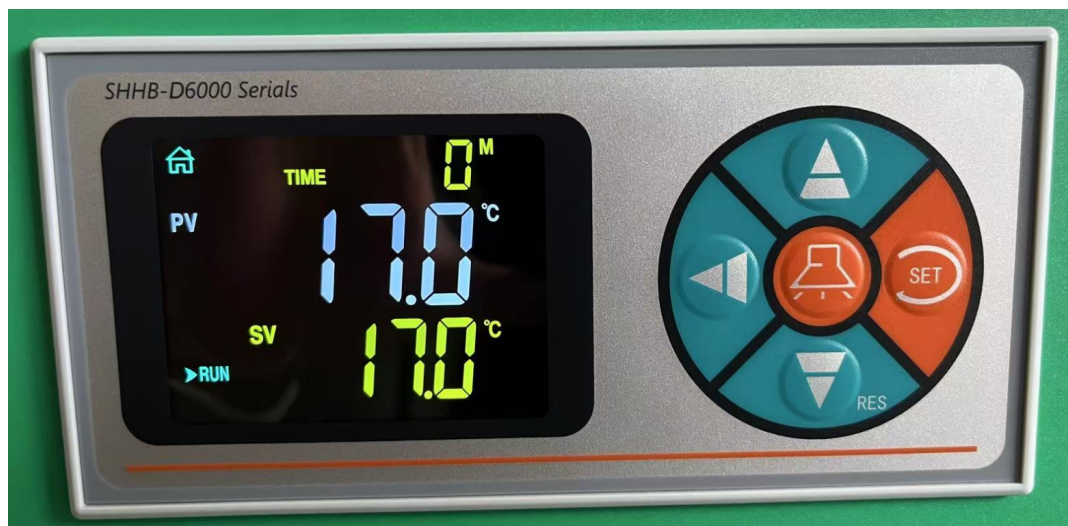

LCD display schematic SEQ Schematische_weergave_LCD_scherm * ARABIC 1

Adjusting the default temperature

By default, the MS Semen Storage Pro 150 is set at 17 °C but you can set it to the precise temperature you require.

Figure 1 shows you the standard menu on the MS Semen Storage Pro 150. The 20 °C is the actual compartment temperature and 17 °C is the set value, to which the unit must be cooled/ heated.

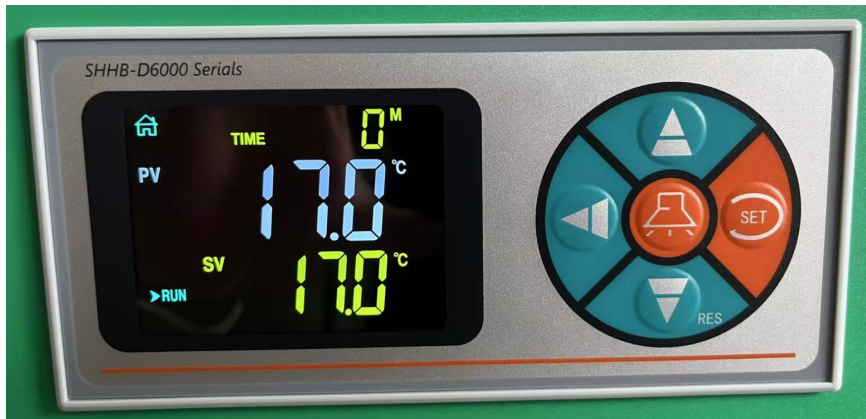
Figure SEQ Figuur * ARABIC 1

The MS Semen Storage Pro 150 has four buttons. They allow you to go through all the steps in the menu. To adjust the default temperature, simply press the button on the extreme right ("Settings") once. You can see the screen that will follow in figure 2.

Figure SEQ Figuur * ARABIC 2

Using the up “▲” and down buttons “▼” you can adjust the default value of 17 °C to the required temperature. If there is no need to change anything here, simply press "settings" again, to return to the start screen.

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